VEVOR[®]

TOUGH TOOLS, HALF PRICE

Technical Support and E-Warranty Certificate

www.vevor.com/support

TOILET HANDRAIL

MODEL:FBL210212

We continue to be committed to provide you tools with competitive price.

"Save Half", "Half Price" or any other similar expressions used by us only represents an estimate of savings you might benefit from buying certain tools with us compared to the major top brands and does not necessarily mean to cover all categories of tools offered by us. You are kindly reminded to verify carefully when you are placing an order with us if you are actually Saving Half in comparison with the top major brands.

VEVOR[®]

TOUGH TOOLS, HALF PRICE

TOILET HANDRAIL

MODEL:FBL210212

NEED HELP? CONTACT US!

Have product questions? Need technical support? Please feel free to contact us:

Technical Support and E-Warranty Certificate
www.vevor.com/support

This is the original instruction, please read all manual instructions carefully before operating. VEVOR reserves a clear interpretation of our user manual. The appearance of the product shall be subject to the product you received. Please forgive us that we won't inform you again if there are any technology or software updates on our product.

Warning-To reduce the risk of injury, user must read instructions manual carefully.


IMPORTANT


1. Don not exceed the weight capacity - 300 lbs max.
2. Use on stable surfaces only.
3. Regularly inspect for signs of wear, damage or loose parts.
4. Clean regularly using a mild detergent and warm water, rinse off any soap residue. After cleaning, dry thoroughly.

Model	FBL210212
Rated Load(lbs / kg)	300 / 136

PART LIST

When unpacking this product, check to make sure the parts listed below are included and carefully inspect for any damage that may have occurred during transit. If any part is missing or damaged, do not attempt to assemble or use the product.

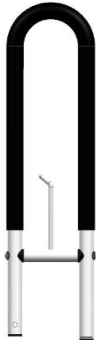NO	PICTURE	PART NAME	QTY	NO	PICTURE	PART NAME	QTY
①		Rail	1	⑥		Bolt	1

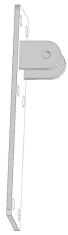
②		Plate	1	⑦		Hex nut	1
③		Bolt Protection Cap	2	⑧		Sleeve Anchor	6
④		Flat washer	2	⑨		Hexagon Keys	1
⑤		Plastic washer	2	⑩		Outer Spanner	2

Installation Procedures


Tools You'll Need:


Step 1: Place the plastic washers on either side of the plate, then put the rail in the center of the washers.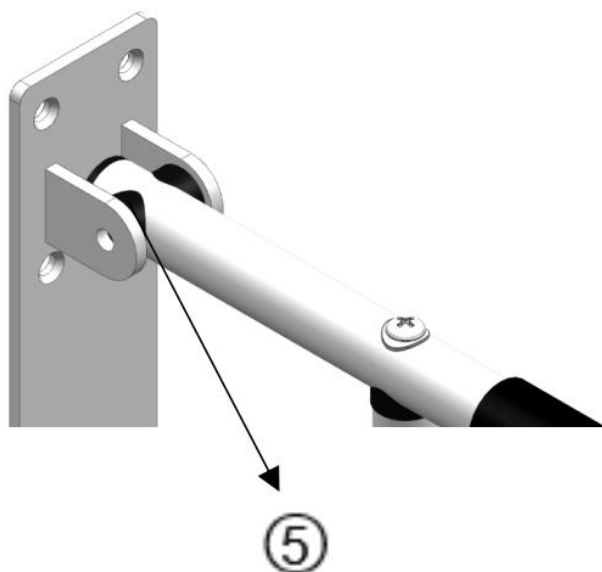


Step 2: Assemble the parts as shown in the diagram.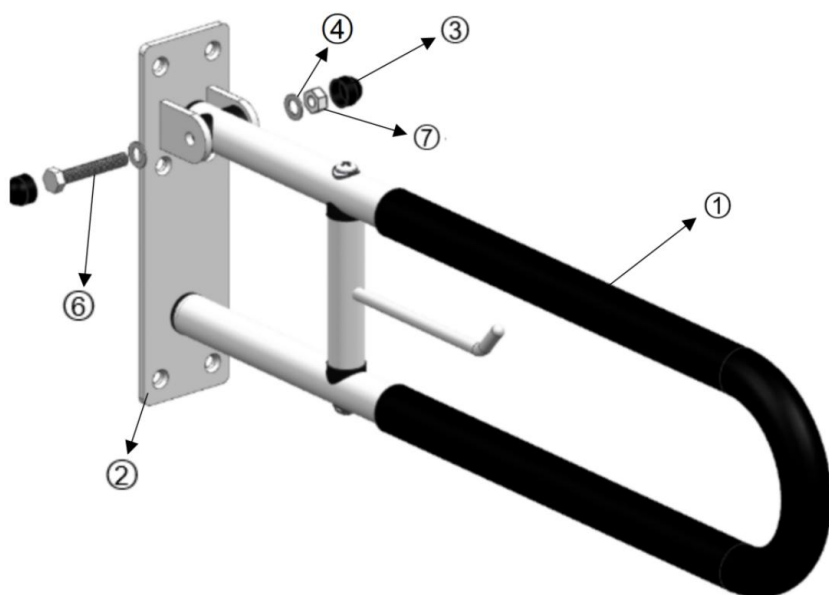


Step 3:Offer the rail up to the wall at the height required, ensuring the fixing plate is straight. Mark the fixing hole locations.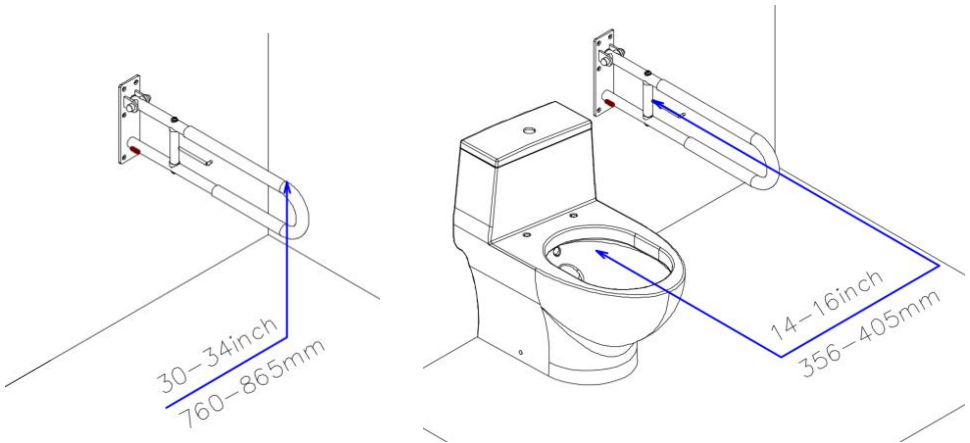


Step 4:Drill the holes in the wall.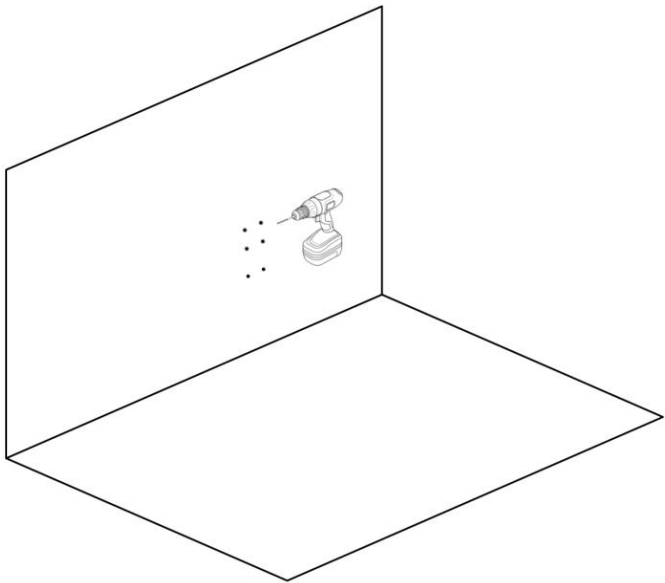

Step 5:Thread the sleeve anchor through the holes in the fixing plate and align them with the holes punched in the wall.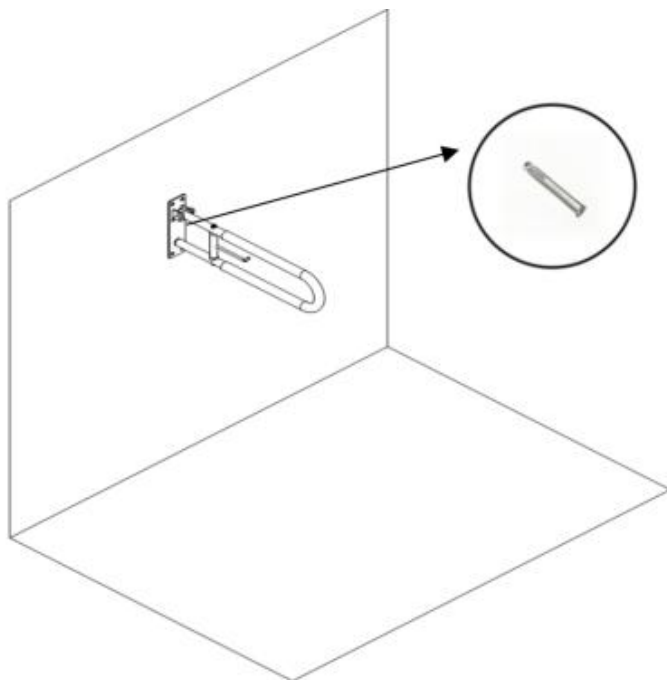

Step 6: Lock the sleeve anchors with a hexagon key to complete the installation.

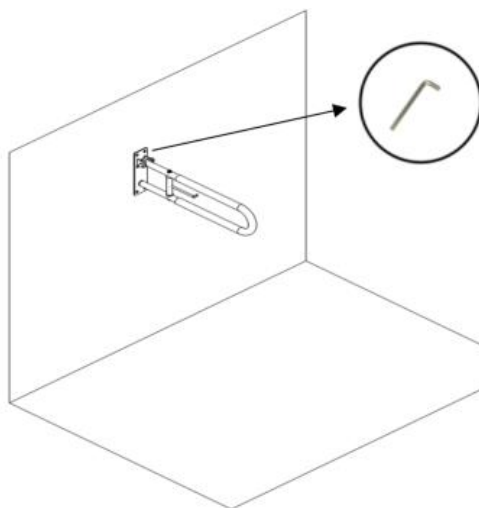Folding Storage

As shown in the picture, the handrail can be folded upwards to save space.

Note: If the handrail cannot be kept in the storage state, it is necessary to slightly tighten the screw at the rotating shaft pointed by the arrow to keep it in the storage state.

Manufacturer: Shanghaimuxinmuyeyouxiangongsi

Address: Shuangchenglu 803nong11hao1602A-1609shi, baoshanqu, shanghai 200000 CN.

Imported to AUS: SIHAO PTY LTD. 1 ROKEVA STREETEASTWOOD NSW 2122 Australia

Imported to USA: Sanven Technology Ltd. Suite 250, 9166 Anaheim Place, Rancho Cucamonga, CA 91730

UK	REP
-----------	------------

YH CONSULTING LIMITED. C/O YH Consulting Limited Office 147, Centurion House, London Road, Staines-upon-Thames, Surrey, TW18 4AX

EC	REP
-----------	------------

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Importerad till AUS: SIHAO PTY LTD. 1 ROKEVA STREETEASTWOOD NSW 2122
till USA: Sanven Technology Ltd. Suite 250, 9166 Anaheim Place, Rancho Cucamonga,
E-CrossStu GmbH