

VEVOR[®]

TOUGH TOOLS, HALF PRICE

Technical Support and E-Warranty Certificate

www.vevor.com/support

THRESHOLD RAMP USER MANUAL

We continue to be committed to provide you tools with competitive price.


"Save Half", "Half Price" or any other similar expressions used by us only represents an estimate of savings you might benefit from buying certain tools with us compared to the major top brands and does not necessarily mean to cover all categories of tools offered by us. You are kindly reminded to verify carefully when you are placing an order with us if you are actually saving half in comparison with the top major brands.

VEVOR[®]

TOUGH TOOLS, HALF PRICE

THRESHOLD RAMP

Model: LD9YV0 /LYIMV0 /Z2UAV0 /GRBRV0 /OVUSV0 /SNPTV0

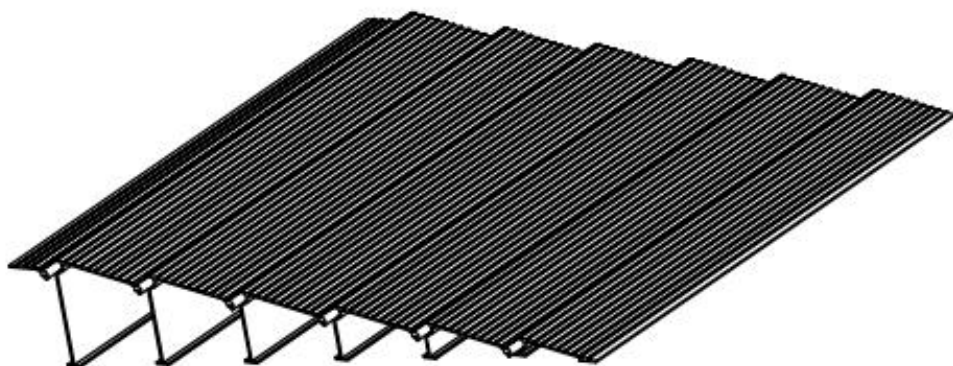

NEED HELP? CONTACT US!

Have product questions? Need technical support? Please feel free to contact us:

Technical Support and E-Warranty Certificate
www.vevor.com/support

This is the original instruction, please read all manual instructions carefully before operating. VEVOR reserves a clear interpretation of our user manual. The appearance of the product shall be subject to the product you received. Please forgive us that we won't inform you again if there are any technology or software updates on our product.

1. Safety Instructions

WARNING:

Carefully read and understand instructions and procedures before installing and using the threshold ramp.

Do not modify the threshold ramp design.

- Do not move a threshold ramp while standing on it.
- Do not use it in bad weather, such as wind, rain, snow, and so on. It is recommended to use the ladder in clear weather.

1. During installation, please wear gloves to prevent hand injuries.
2. Do not stack items or allow children to play on the ramp.
3. Regularly check the structure and fasteners of the ramp to ensure its stability and safety.
4. If the surface of the ramp is worn or damaged, please replace or repair it in time.

2. Maintenance Guide

1. Place the product on a sturdy, secure, accessible, and level starting point. The minimum weight capacity of the starting point must be equal to that of the ramp (800 lbs). Do not exceed the maximum weight capacity of the product.
2. Ensure that the product is securely flush against the starting point. If a safe, accessible, and level starting point cannot be found, or if the product does not fit securely against a higher starting point, do not use the ramp.
3. Do not tamper with, attempt to repair, or modify any part of the product. Exercise caution to avoid pinch points during assembly and handling.
4. If using the product with other mobility equipment, refer to the user guide of the equipment to understand the acceptable usage guidelines, including appropriate slope, incline, and chair direction. Do not exceed the

recommendations of the mobility equipment.

5. Before each use, carefully inspect the fasteners on the ramp to ensure they are not loose. Tighten any loose fasteners if necessary.
6. When using the ramp, ensure it is securely flush against the starting point to ensure safety.
7. Before each use, clean the ramp surface of ice, snow, dust, leaves, and other debris. Remove accumulated ice and snow, and sweep the surface clean. For dust and debris, use a soft bristle brush and non-alkaline detergent to clean the ramp surface, then rinse thoroughly.
8. The ramp surface may become slippery in wet conditions. Walk with caution and pay attention to your footing to prevent slipping.
9. If the ramp will not be used for an extended period, store it in a dry and well-ventilated area away from direct sunlight.

SAVE THESE INSTRUCTIONS







3. Product Description and Main Features

The threshold interlocking aluminum alloy ramp is made of high-quality aluminum alloy extrusion. This product is durable and provides convenience for people. It is lightweight, portable, stable, and durable and features an interlocking design for easy installation and storage.

➤ **Main Features:**


1. **Material:** High-strength aluminum alloy, extruded.
2. **Surface:** Anti-slip treatment, deburring.
3. **Interlocking design:** Each section's interface is precisely designed; align each section's interface and push them together for installation.
4. **Load limit:** Up to 800 lbs.
5. **Height optional:** suitable for thresholds of different heights.

4. Product Parameter


| Item | Model | Picture | Height | Load Limit | N.W | G.W |
|------|--------|---|--------|------------|--------|---------|
| 1 | LD9YV0 |  | 1inch | 800lbs | 1.9kg | 2.6kg |
| 2 | LYIMV0 |  | 2inch | 800lbs | 3.6kg | 4.4kg |
| 3 | Z2UAV0 |  | 3inch | 800lbs | 5.4kg | 6.25kg |
| 4 | GRBRV0 |  | 4inch | 800lbs | 7.35kg | 8.35kg |
| 5 | OVUSV0 |  | 5inch | 800lbs | 9.4kg | 10.5g |
| 6 | SNPTV0 |  | 6inch | 800lbs | 11.7kg | 12.55kg |

5. Installation Guide







- (1) Before starting the installation, please wear gloves to protect your hands.
- (2) Choose an appropriate installation location, ensuring that the ground is level and firm.
- (3) Align and interlock each section of the ramp together until all parts are even.
- (4) Starting with the 1" extension, install the threshold extensions in the order shown (FIG. 1) until the final height is reached. Bring the threshold extensions together at an approximate 45° angle, then rotate them into place after the bulb is engaged in the slot.

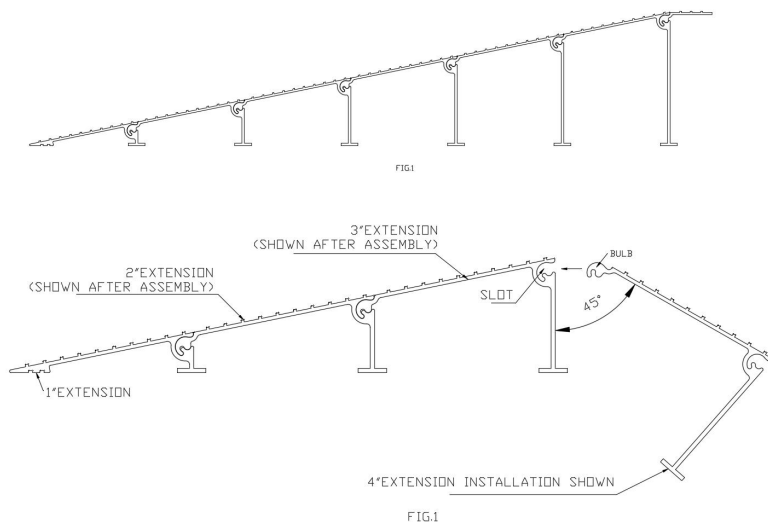

- (5) Use the provided screws to tighten each section of the ramp to ensure that the ramp does not slip out or make noise during use.

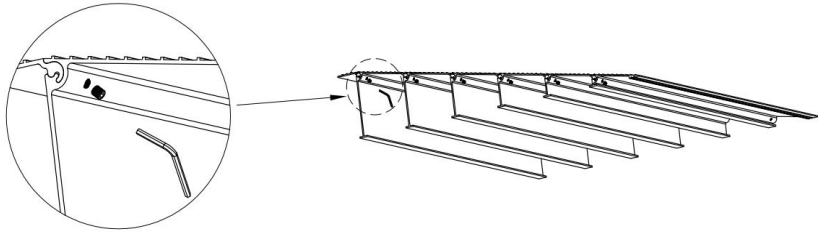

- (5) After installation, check whether the ramp is stable and tightly attached to the ground.
- (6) When using the ramp, push the equipment up and down the ramp steadily, avoiding sharp turns or speeding.

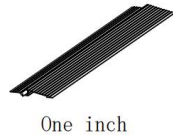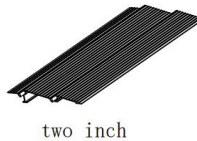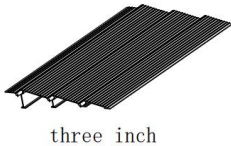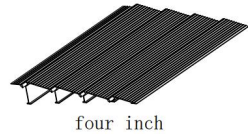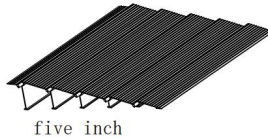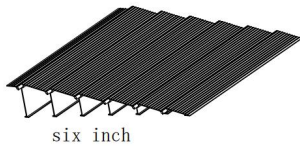

Manufacturer: Shanghaimuxinmuyeyouxiangongsi

Address: Shuangchenglu 803nong11hao1602A-1609shi, baoshanqu, shanghai 200000 CN.

Imported to AUS: SIHAO PTY LTD. 1 ROKEVA STREET EASTWOOD NSW 2122 Australia

Imported to USA: Sanven Technology Ltd. Suite 250, 9166 Anaheim Place, Rancho Cucamonga, CA 91730



E-CrossStu GmbH
Mainzer Landstr.69, 60329 Frankfurt am Main.



YH CONSULTING LIMITED.
C/O YH Consulting Limited Office 147, Centurion House, London Road, Staines-upon-Thames, Surrey, TW18 4AX

VEVOR[®]

TOUGH TOOLS, HALF PRICE

Technical Support and E-Warranty Certificate

www.vevor.com/support