VEVOR[®]

TOUGH TOOLS, HALF PRICE

Technical Support and E-Warranty Certificate www.vevor.com/support

COMMODE

MODEL:FBL760101

We continue to be committed to provide you tools with competitive price.

"Save Half", "Half Price" or any other similar expressions used by us only represents an estimate of savings you might benefit from buying certain tools with us compared to the major top brands and does not necessarily mean to cover all categories of tools offered by us. You are kindly reminded to verify carefully when you are placing an order with us if you are actually saving half in comparison with the top major brands.

VEVOR®

TOUGH TOOLS, HALF PRICE

COMMODE

MODEL:FBL760101

NEED HELP? CONTACT US!

Have product questions? Need technical support? Please feel free to contact us:

Technical Support and E-Warranty Certificate
www.vevor.com/support

This is the original instruction, please read all manual instructions carefully before operating. VEVOR reserves a clear interpretation of our user manual. The appearance of the product shall be subject to the product you received. Please forgive us that we won't inform you again if there are any technology or software updates on our product.

WARNING:

Read the manual completely to avoid harm, product damage, or warranty avoidance. Seek professional help if you encounter discomfort, pain or worsening symptoms. If necessary, use supervision in order to assemble and use this product correctly and safely. Do not exceed the weight limit.

Regularly check for secure assembly and contact support if in doubt. Ensure the product's legs are even and locked before use, and discontinue usage if there are signs of wear, damage, or missing parts. The product is to be used on flat, stable surfaces. Do not disassemble if not necessary, as it might compromise the longevity of the product. Do not modify the product unless specified in the manual.

Physiotherapist

1. Before first use, adjust the height and the width of the rail to your toilet.
2. Back up to the toilet with your assistive device (if available) until the back of your legs are touching the toilet.
3. Reach back for the rail, hold firmly, and slowly lower yourself into a seated position on the toilet.
4. To return to a standing position, place your hands on the rails firmly and push yourself up into a standing position.
5. Once you are standing, gradually let go of the rails, and only then, grab the handle of your assistive device (if available).

IMPORTANT

1. Don't exceed the maximum weight capacity.
2. Use on stable surfaces only.
3. Regularly inspect for signs of wear, damage, or loose parts.
4. Clean regularly using a mild detergent and warm water, rinse off any soap residue. After cleaning, dry thoroughly.
5. Before use, confirm with your healthcare provider that the product meets your specific needs. The following guidelines are not a substitute for medical advice.

SAVE THESE INSTRUCTION

PART LIST

When unpacking this product, check to make sure the parts listed below are included and carefully inspect for any damage that may have occurred during transit. If any part is missing or damaged, do not attempt to assemble or use the product.

NO	PICTURE	NAME	QTY	NO	PICTURE	NAME	QTY
①		Leg Tube	4	⑥		Seat Frame	1
②		Backrest Connecting Tube	2	⑦		Pail	1
③		Backrest	1	⑧		Foot Mats	2
④		Left Handrail	1	⑨		Hex Wrench	1
⑤		Right Handrail	1				

PRODUCT PAREMETERS

Model	FBL760101
Seat Measures (L×W)	415×415mm/16.3"×16.3"
Seat Height	490-640mm/19.3"-25.2"
Seat Rated Load	159kg / 350lbs

INSTALLATION AND OPERATION INSTRUCTIONS

1. Installation Procedures

1

As shown in the picture: (Installation of the leg tube) Press the marble to insert the foot tube into the mouth of the main frame tube, and the marble pops out and snaps into place.

2

As shown in the figure: (Installation of the backrest) Before installing the backrest, press the marble and insert the backrest connecting tube into the installation position.

3

As shown in the picture: (Installing the handrail) Insert the handrail into the main frame, screws with spacers to lock the handrail.

4

As shown in the picture: press and hold the handrail button to turn your handrail by 90°.

5

As shown in the picture: the installation is complete.

Zhongshan Bliss Medical Instrument Co., Ltd

Address: 3-6F & 7F-1, No. 63, Tongxing East Road, Yumin Community
Xiaolan, Zhongshan, Guangdong, 528414, CHINA

UK

REP

Pooledas Group Ltd

Unit 5 Albert Edward House,

The Pavilions Preston, United Kingdom

pooledas123@gmail.com

01772418127

EC

REP

SHUNSHUN GmbH

Römeräcker 9 Z2021, 76351

Linkenheim-Hochstetten, Germany

euvertreter@gmail.com

+49 1727041930

Made In China

VEVOR[®]

TOUGH TOOLS, HALF PRICE

Technical Support and E-Warranty Certificate

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