VEVOR[®]

TOUGH TOOLS, HALF PRICE

Technical Support and E-Warranty Certificate

www.vevor.com/support

FOLDING THRESHOLD-RAMP

USER MANUAL


We continue to be committed to provide you tools with competitive price.

"Save Half", "Half Price" or any other similar expressions used by us only represents an estimate of savings you might benefit from buying certain tools with us compared to the major top brands and does not necessarily mean to cover all categories of tools offered by us. You are kindly reminded to verify carefully when you are placing an order with us if you are actually saving half in comparison with the top major brands.

VEVOR®

TOUGH TOOLS, HALF PRICE

FOLDING THRESHOLD-RAMP

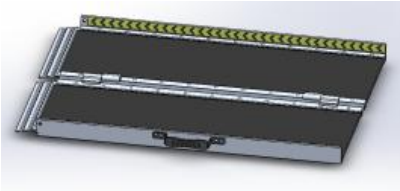TYPE A	TYPE B
 A black folding threshold-ramp with a silver metal frame and a green and yellow striped safety strip along the top edge.	 A black folding threshold-ramp with a silver metal frame and a green and yellow striped safety strip along the top edge, similar to Type A but with a different internal structure.
Model: 21YF5QCV0 / 19UQ9FV0 / 1YAZTIV0 / 12YUIHV0 / 1JQ8CRV0	Model: CUEI5QV0 / 13E5LEV0 / 17MST1V0

NEED HELP? CONTACT US!

Have product questions? Need technical support? Please feel free to contact us:

Technical Support and E-Warranty Certificate
www.vevor.com/support

This is the original instruction, please read all manual instructions carefully before operating. VEVOR reserves a clear interpretation of our user manual. The appearance of the product shall be subject to the product you received. Please forgive us that we won't inform you again if there are any technology or software updates on our product.

1. Important Safety Instructions

WARNING:

1. Carefully read and understand instructions and procedures before installing and using the threshold ramp. Prior to assembly and use, read and follow all instructions, warnings, and labels.

	Warning - To reduce the risk of injury, user must read instructions manual carefully. Read and understand this instruction manual prior to using this product. Failure to do so may result in serious injury or death.
	Warning- Be sure to wear gloves when using this product.

2. This product is mainly used by users to push wheelchairs, strollers, and carry large objects across thresholds.

- (1) AVOID SERIOUS INJURY OR DEATH
- (2) DO NOT Exceed the weight capacity or load height of the ramp.
- (3) Use Ramps on a stable and level surface only.
- (4) It is strictly prohibited for users to drive the wheelchair alone on the ramp.
- (5) Only use the ramp with a qualified helper and always use a lap belt.
- (6) When carrying heavy objects through a ramp, be sure to have enough auxiliary personnel to assist in ensuring that people or goods do not tip over and cause damage to people or goods when using the ramp.

3. Before use, refer to your mobility equipment's owner guide for acceptable ramp usage, including proper slope/incline, chair direction; never exceed its recommendations.

4. Do not use the ramp if it is damaged or unstable.

5. Do not tamper with, attempt to repair, or modify any portion of the ramp.


6. Use caution when folding and unfolding the ramp to avoid all potential pinch points.

7. Use it only if the secure,unobstructed,level landings can support,at minimum,the same weight capacity as the ramp(800lbs.)
8. The ramp surface may be slippery in wet conditions;use caution if the surface is wet as traction may be reduced
9. Do not use the ramp if its surface is covered with ice,snow,dirt,leaves,or other debris.Remove all accumulations and sweep the surface clean before use.
10. Improper use of the ramp could result in serious injury.
11. Always exercise caution when handling,installing,and or using the ramp.
12. Casual placement may result in accidental injury due to children's climbing or playing; or the product may fall off and cause major accidents.During walking, the product may shift or dislocate due to the reaction of your walking.Please be careful.
13. Please do not disassemble and reinstall this product, Otherwise, it may cause serious accidents

SAVE THESE INSTRUCTIONS


2.Before use

1. Use it only if sufficient maneuvering room is available at both ends of the ramp to ensure safe travel when entering and exiting the ramp.
2. Unpack the product from the packaging and pull out the locking pin to unfold the product. Ensure that the landings the ramp will be placed on are substantial and of sound construction.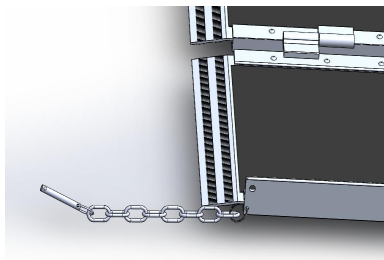

3.Connecting board:

Please place the product firmly on the steps and make sure it is stable before use.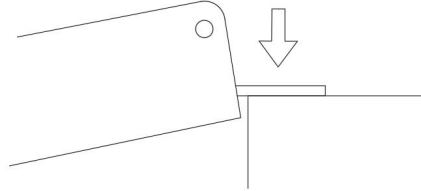

4.Before each use,check the ramp for worn,loose,or damaged parts.If you find any of these conditions,do not use it.


3.How to USE

Step1:Unpack the product from the packaging .

Step2:Remove the latch from the latch hole and unlock the product

Step3:Unfold the product.

(TYPE A)
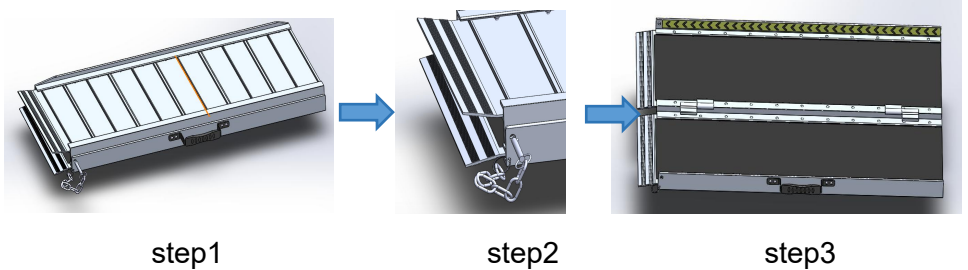


(TYPE B)

Step1:Unpack the product from the packaging .



Step2:Remove the latch from the latch hole and unlock the product

Step3~Step7:Unfold the product.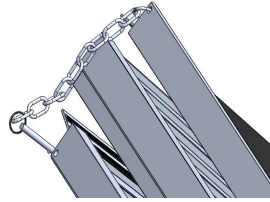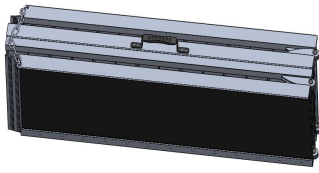

step1



step2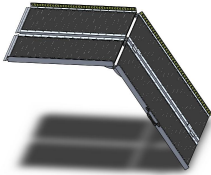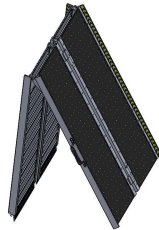

step5

step4


step3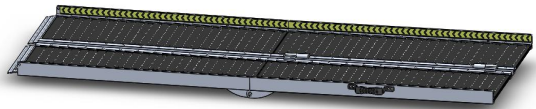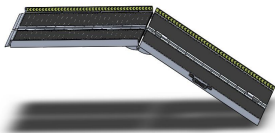

step6


step7

Step8: Please put this product firmly on the step. Choose a ramp of appropriate length based on the recommended height. The load-bearing capacity of the ramp should not exceed 800lbs. Before use, please make sure that this product can not be shifted.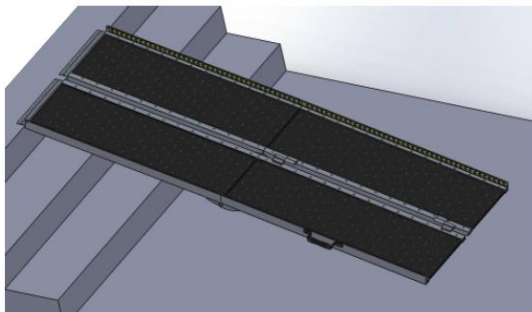

Notes:

1. When the latch is not removed from the latch hole, please do not forcefully unfold the product.
2. Fold the products by reversing the order in which they were expanded.
3. When using a wheelchair on this product, please be sure to move it slowly from a low position to a higher one. And when you move it from a high place to a lower place, it may cause a fall for the wheelchair user. In addition, a sudden push or sudden braking of a wheelchair may result in an accident. Please be slow when pushing the wheelchair.

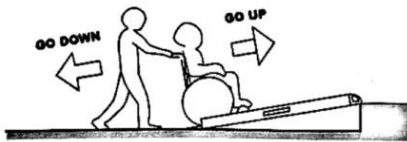Right(✓)

Wrong (X)

4.Product Parameter

Item	Model	Length	Appropriate height from steps to ground	Max Weight	Material	Note
1	1YF5QCV0	24inch	100-150mm	800lbs	Aluminum Al10y	TYPE A
2	19UQ9FV0	36inch	150-200mm	800lbs	Aluminum Al10y	
3	1YAZTIV0	48inch	200-250mm	800lbs	Aluminum Al10y	
4	12YUIHV0	60inch	250-300mm	800lbs	Aluminum Al10y	
5	1JQ8CRV0	72inch	300-360mm	800lbs	Aluminum Al10y	
6	13E5LEV0	84inch	360-420mm	800lbs	Aluminum Al10y	TYPE B
7	17MST1V0	96inch	420-500mm	800lbs	Aluminum Al10y	
8	CUEI5QV0	120inch	500-600mm	800lbs	Aluminum Al10y	

5.Maintenance Guide

1. Place the product on a sturdy, secure, accessible, and level starting point. The minimum weight capacity of the starting point must be equal to that of the ramp (800 lbs). Do not exceed the maximum weight capacity of the product.
2. Ensure that the product is securely flush against the starting point. If a safe, accessible, and level starting point cannot be found, or if the product does not fit securely against a higher starting point, do not use the ramp.
3. Do not tamper with, attempt to repair, or modify any part of the product. Exercise caution to avoid pinch points during assembly and handling.
4. If using the product with other mobility equipment, refer to the user guide of the equipment to understand the acceptable usage guidelines, including appropriate slope, incline, and chair direction. Do not exceed the recommendations of the mobility equipment.
5. When using the ramp, ensure it is securely flush against the starting point to ensure safety.
6. Before each use, clean the ramp surface of ice, snow, dust, leaves, and other debris. Remove accumulated ice and snow, and sweep the surface clean. For dust and debris, use a soft bristle brush and non-alkaline detergent to clean the ramp surface, then rinse thoroughly.
7. The ramp surface may become slippery in wet conditions. Walk with caution and pay attention to your footing to prevent slipping.
8. If the ramp will not be used for an extended period, store it in a dry and well-ventilated area away from direct sunlight.
9. Do not use thinner, gasoline or other volatile organic solvents to clean the product.
10. Do not scrub this product with a hard brush or hard sponge.

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YH CONSULTING LIMITED.
C/O YH Consulting Limited Office 147, Centurion House, London Road, Staines-upon-Thames, Surrey, TW18 4AX

VEVOR[®]

TOUGH TOOLS, HALF PRICE

Technical Support and E-Warranty Certificate

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Importerad till AUS: SIHAO PTY LTD. 1 ROKEVA STREETEASTWOOD NSW 2122 Australien
Importerad till USA: Sanven Technology Ltd. Suite 250, 9166 Anaheim Place, Rancho Cucamonga, CA 91730
E-CrossStu GmbH