

VEVOR[®]

TOUGH TOOLS, HALF PRICE

Technical Support and E-Warranty Certificate
www.vevor.com/support

HEARING AID

MODEL:GM-912

We continue to be committed to provide you tools with competitive price.

"Save Half", "Half Price" or any other similar expressions used by us only represents an estimate of savings you might benefit from buying certain tools with us compared to the major top brands and does not necessarily mean to cover all categories of tools offered by us. You are kindly reminded to verify carefully when you are placing an order with us if you are actually saving half in comparison with the top major brands.

VEVOR®

TOUGH TOOLS, HALF PRICE

MODEL: GM-912



NEED HELP? CONTACT US!

Have product questions? Need technical support? Please feel free to contact us:

Technical Support and E-Warranty Certificate
www.vevor.com/support

This is the original instruction, please read all manual instructions carefully before operating. VEVOR reserves a clear interpretation of our user manual. The appearance of the product shall be subject to the product you received. Please forgive us that we won't inform you again if there are any technology or software updates on our product.



Warning-To reduce the risk of injury, user must read instructions manual carefully.



CORRECT DISPOSAL

This product is subject to the provision of European Directive 2012/19/EC. The symbol showing a wheelie bin crossed through indicates that the product requires separate refuse collection in the European Union. This applies to the product and all accessories marked with this symbol.

Products marked as such may not be discarded with normal domestic waste, but must be taken to a collection point for recycling electrical and electronic devices



To minimize hazards to health and the environment at the end of this product's life, laws dealing with Waste Electrical and Electronic Equipment (WEEE) and The Waste Battery Directive require you to dispose of this product at a suitable collection facility where it will be sent in order to remove the batteries and for appropriate recycling. Please contact your local authorities for more details on recycling and safe disposal of these in your area.



Compliance is a EC & UK security certification.

**WARNING**

Move away from any potential source of ignition immediately if the battery leaks or emits an unusual odor. When you notice unusual smell, heat, discoloration, deformation, or any other unusual situations in the process of using, charging, or storing batteries, you should immediately remove them from the device or charger and stop using them.

**WARNING**

This product are not for use of persons under the age of 18.

FCC INFORMATION

CAUTION: Changes or modifications not expressly approved by the party responsible for compliance could void the user's authority to operate the equipment!

This device complies with Part 15 of the FCC Rules. Operation is subject to the following two conditions:

- 1) This product may cause harmful interference.
- 2) This product must accept any interference received, including interference that may cause undesired operation.

WARNING: Changes or modifications to this product not expressly approved by the party responsible for compliance could void the user's authority to operate the product.

Note: This product has been tested and found to comply with the limits for a Class B digital device pursuant to Part 15 of the FCC Rules, These limits are designed to provide reasonable protection against harmful interference in a residential installation.

This product generates, uses and can radiate radio frequency energy, and if not installed and used in accordance with the instructions, may cause harmful interference to radio communications. However, there is no guarantee that interference will not occur in a particular

installation. If this product does cause harmful interference to radio or television reception, which can be determined by turning the product off and on, the user is encouraged to try to correct the interference by one or more of the following measures.

- Reorient or relocate the receiving antenna.
- Increase the distance between the product and receiver.
- Connect the product to an outlet on a circuit different from that to which the receiver is connected.
- Consult the dealer or an experienced radio/TV technician for assistance.

The device has been evaluated to meet general RF exposure requirement. The device can be used in portable exposure condition without restriction.

IMPORTANT SAFEGUARDS

Thank you for using this product. In order to make sure that you can operate the machine correctly,

read this instruction carefully before operation and keep it properly for future reference. Please be sure to read the precautions and safety rules on this page to ensure your safe use.

This manual will outline safety warnings and precautions, operating, maintenance and cleaning. The warnings and instructions reviewed in this manual cannot cover all possible conditions and situations that may occur. Caution and common sense are not built into this product, since we believe that the uses will comply with these codes. If you have any problems, please call us. Do not try to fix the problems yourself, or it will cause injury or property damage.

Please read ALL the instructions before using your machine.

1. If the battery is low, please charge it in time.
2. Do not let children, minors, and pets get close to or touch the machine.
3. Do not immerse the appliance in water or any other liquid.
4. Do not disassemble the product.
5. Using any accessories or attachments that are

not provided or recommended by the manufacturer may cause injury.

6. Please start to use after the collect is fixed well.

7. Do not leave the product hanging over the edge of table or counter or let it come into contact with any hot surfaces.

8. Do not place the product on or near hot air, electric burners or heaters.

9. Do not apply this unit to any other purposes than the indicated usage.

10. KEEP IT OUT OF CHILDREN'S REACH.

11. DO NOT CLEAN IT WITH ANY ABRASIVE MATERIAL.

12. Please clean it regularly, at least once a month.

13. This appliance is not intended for use by persons (including children) with reduced physical, sensory or mental capabilities, or lack of experience or knowledge, unless they have been given supervision or instruction concerning the use of the appliance by a person responsible for their safe.

BEFORE USING

1. Before using the appliance for the first time, peel off any promotional and packaging materials and check that the product is clean and dust-free.
2. Please dispose of all plastic bags carefully and keep them away from children and pets.
3. Check all components provided according to the list in this manual.

PRODUCT PARAMETERS

Model	GM-912
Charging port	Type-C
Power In put	DC5V 1A

Note: Please use the correct power source voltage.

Note: Before first use, the earphones need to be placed in the charging compartment to charge for 10-12 hours.

BEFORE WEARING



- It is important to make sure your ears are clean before wearing hearing aids. Take a cotton swab and clean any wax around your outer ear.

Attention: DO NOT push the cotton swab down into your inner ear!

Only clean the outer part of your ears!

If there is too much wax build-up in your ear, it will block the sound from your hearing aids.

HOW TO WEAR



Choose an ear dome size that you think will fit your ears. You may need to try different size until you find the perfect fit. Carefully slip the domes off the hearing aid, then slip the new size dome on.



Insert your hearing aid into ear canal. It should feel snug, secure, and comfortable. If not, try a different size dome.

Be sure to choose the best fitting ear dome for your ears! Everyone's ears are different! If the ear dome are too small or too loose for your ear, you will most likely hear feedback of whistling. Always keep your hearing aids turned off until you have them in your ears to prevent feedback.

TURN ON


Put the hearing aid switch to "N", waiting for 10 seconds to start working.





When wearing hearing aids, please do not cover the microphone above.

TURN OFF

Put the hearing aid switch to "O", the hearing aids will turn off.

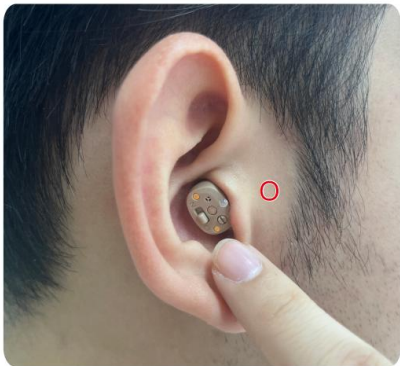

VOLUME CONTROL

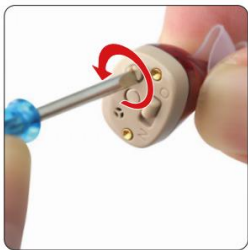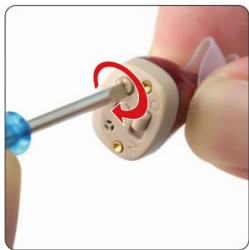

Turn up the volume


Turn down the volume

As shown in the figure: Take out the tuning stick to adjust the volume.

Rotate clockwise = turn up the volume

Rotate counterclockwise = turn down the volume

Attention: The first time you use a hearing aid to adjust the volume, you should start the trial adjustment from the minimum volume.

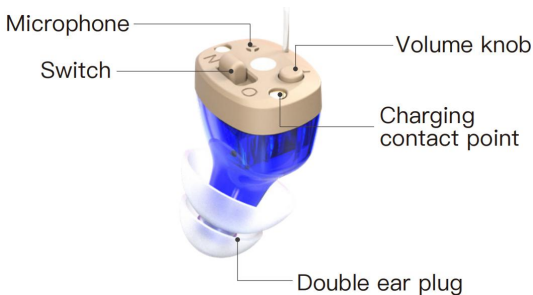

CHARGING

L+ Green Bars: Left hearing aid is charging

R+ Green Bars: right hearing aid is charging

The number [0-100] indicates the level of charge of case has left . As long as this number above 0, you can place your hearing aids in the case to recharge them.

Attention: It is recommended to recharge the case (using the USB cable included) any time the case is below 30%.



Charging



Charging complete

1. **Charging Time:** For the first charge, it is recommended to charge for 10-12 hours to ensure the battery is fully activated for optimal performance.
2. **Use Original Charge cable:** Always use the provided original cable to ensure a secure connection and avoid charging efficiency issues.
3. 2-3 hours of charging time can make your hearing aid fully charged.
4. **Temperature Range:** Avoid charging in extreme temperature conditions. Maintain a charging Celsius to safeguard battery health.environment within the range of 5-35 degrees.
5. **Long-Term Storage:** If hearing aids will not be used for an extended period, power them off and

store them in a dry place to prevent moisture-related issues and ensure device performance.

21-DAY WEAR CHART

THE JOURNEY TO LISTENING

Unlike vision correction, which simply wears glasses, hearing correction is a rather detailed process. Once you start your hearing correction journey, you will experience countless "new sounds", and your brain needs to adapt for a period of time. In just a few weeks, you will be shocked by the sound that you have never experienced before. If you have any questions, we will serve you wholeheartedly.

● THE FIRST WEEK

Some sounds may be too noisy when wearing a hearing aid for the first time, such as the humming of the refrigerator, the wrinkling of paper, or the sound of your footsteps. This is normal, and your brain is re-learning how to prioritize these sounds.

Be patient!

Patient

At the beginning, wear the hearing aid for 30 minutes a day. Each day, try increasing your wearing time by another 30 minutes.

Exercise

Listen carefully to a song or watch a movie you familiar with. Embrace being able to hear again!

● THE SECOND WEEK

The more you wear the hearing aid, the more you can adapt to the different sounds in different environments. You can even try telling the people around you to not speak so loudly to you any more. Communication becomes much easier now.

● THE THIRD WEEK

After a few weeks, you will feel much more energetic because you no longer need to spend so much energy trying to hear what others say! The sound will become cleaner and clearer. Listening will come naturally. You will once again enjoy the sounds that surround you in your life.

Participate

Have a meal in a restaurant with friends, watch a movie in a theater, join a family gathering, and no longer have to worry about missing out on memories!

TROUBLESHOOTING

1. There is no sound when wearing? If there is no sound:

- Check to see if the hearing aid is switched On.
- Make sure the hearing aid is fully charged.
- Check the volume level to see if it is adjusted to suit your hearing.
- Make sure your ears are clean and there is nothing blocking your ear canal.

Note: If the hearing aid is silent or the sound becomes low due to the blockage of the sound outlet, please clean the sound outlet

2. If you hear constant feedback or a high sharp sound:

You may need to try different sized ear dome. Feedback usually means that the dome is too small, and the sound is escaping your ear canal. The ear dome should be big enough to prevent feedback, but also feel comfortable and secure in your ear.

TIPS

1. After using for more than 60 minutes, it is recommended to remove the headphones and let the ears rest fully before using them again.
2. Please test the volume of the earphones after powering on and avoid putting them directly into your ears.
3. Avoid using headphones during sports, such as swimming, running, skipping, basketball, football, etc.
4. Strong magnetic or electric fields and high noise will damage the product

Note: Before first use, the earphones need to be placed in the charging compartment to charge for 10-12 hours.

CLEANING AND STORAGE

1. Long-Term Storage: If hearing aids will not be used for an extended period, power them off and store them in a dry place to prevent moisture-related issues and ensure device performance.
2. Wipe with a soft dry cloth and clean with a soft bristle brush.
3. Please keep accessories and products together.

Hong yu kang Technology (Guangdong) Co., Ltd.

Address: Room 301,302,303,3rd Floor, Building
12, Fuqiao Second District, HepingCommunity,
Fuhai Street, Baoan District,
Shenzhen,Guangdong,China,518103

Imported to AUS: SIHAO PTY LTD. 1 ROKEVA
STREETEASTWOOD NSW 2122 Australia

Imported to USA: Sanven Technology Ltd. Suite
250, 9166 Anaheim Place, Rancho Cucamonga,
CA 91730



YH CONSULTING LIMITED.

C/O YH Consulting Limited
Office 147, Centurion House,
London Road, Staines-upon-
Thames, Surrey, TW18 4AX



E-CrossStu GmbH
Mainzer Landstr.69,
60329 Frankfurt am Main.

VEVOR[®]

TOUGH TOOLS, HALF PRICE

**Technical Support and E-Warranty
Certificate www.vevor.com/support**