# **VEVOR<sup>®</sup>**

## **TOUGH TOOLS, HALF PRICE**

Technical Support and E-Warranty Certificate

[www.vevor.com/support](http://www.vevor.com/support)

### **STANDARD ASSISTED DRIVING**

**MODEL:FBL810277**

We continue to be committed to provide you tools with competitive price.

"Save Half", "Half Price" or any other similar expressions used by us only represents an estimate of savings you might benefit from buying certain tools with us compared to the major top brands and does not necessarily mean to cover all categories of tools offered by us. You are kindly reminded to verify carefully when you are placing an order with us if you are actually saving half in comparison with the top major brands.

# VEVOR®

TOUGH TOOLS, HALF PRICE

STANDARD  
ASSISTED DRIVING

MODEL: FBL810277

## NEED HELP? CONTACT US!

Have product questions? Need technical support? Please feel free to contact us:

**Technical Support and E-Warranty Certificate**  
**[www.vevor.com/support](http://www.vevor.com/support)**

This is the original instruction, please read all manual instructions carefully before operating. VEVOR reserves a clear interpretation of our user manual. The appearance of the product shall be subject to the product you received. Please forgive us that we won't inform you again if there are any technology or software updates on our product.

## INSTRUCTIONS

### **Preface**

First of all, let us express our heartfelt thanks to our customers who have purchased our equipment. This manual is for customer service to make better use of this equipment. We hope the company's products and services will bring you the same courtesy.

### **Pay attention**

This product must be used in strict accordance with the use method of this manual, misuse of any other methods caused by misuse. The Company does not give any guarantee of any kind, including (but not limited to) the marketability offered to the Company for a particular purpose. Implied warranties with fitness.

The company accepts no liability. The Company shall not be liable for any errors contained in this material, or for any accidental or consequential damages resulting from the provision, actual performance or use of this manual.

This manual contains proprietary information protected by patent law. All rights reserved. Without the written consent of the Company, this manual is not allowed to be photocopied, copied, or translated any part into another language.

The content contained in the manual may be changed without the user's notice.

### **Manufacturer's liability**

The Company shall be held responsible for the safety and reliability of the performance of the equipment only if the assembly operation, returning, improvement and maintenance are carried out by personnel approved by the Company, the electrical equipment concerned conforms to national standards, and the equipment is used in accordance with the operating instructions.

### **Guarantee**

Manufacturing process and raw materials: The Company guarantees that the Equipment (except parts) are in regular use and repair condition, free of manufacturing process and raw materials faults for one year from the date of shipment. None of our inspections is perfect.

## PRECAUTIONS

### Precautions for use

#### Warning:

1. If the equipment feels loose, falls or is damaged, please contact the company in time;
2. Keep work area clean and well lit. Cluttered or dark areas invite accidents.
3. Keep children and bystanders away while operating, distractions will cause you to lose control.
4. During normal use, please use with the accompaniment of family or friends.
5. Reasonably use according to one's own situation, and do not overuse when the body is not in a timely manner.
6. BEFORE USE, PLEASE CONFIRM WHETHER THE COMPONENTS ARE COMPLETE, WHETHER THE BRAKE IS SENSITIVE, AND ASSEMBLE THE ENTIRE PRODUCT IN STRICT ACCORDANCE WITH THE INSTRUCTIONS.
7. CLEANING PRODUCTS NEED TO BE WASHED WITH WATER AND DRY IN TIME. IF THE SURFACE COATING OF THE PRODUCT HAS SCRATCHES AND FALLS OFF, REPAIR MATERIALS NEED TO BE USED IN TIME TO ACHIEVE THE BEST ANTI-RUST EFFECT.
8. PLEASE ENSURE THAT THE REAR WHEEL OF THE PRODUCT IS LOCKED TO AVOID SLIDING WHEN THE USER STAYS IN THE SLOPING ENVIRONMENT.
9. PLEASE ENSURE THAT THE FOLDING MECHANISM OF THE SEAT PLATE IS LOCKED WHEN USING THIS PRODUCT.

### SAVE THESE INSTRUCTIONS

## PRODUCT PARAMETERS

|                          |                           |
|--------------------------|---------------------------|
| Product Name             | Standard assisted driving |
| model                    | FBL810277                 |
| Load bearing capacity/lb | 350lbs                    |

## PRODUCT COMPONENTS

|   |                                                                                   |           |   |   |                                                                                   |                            |   |
|---|-----------------------------------------------------------------------------------|-----------|---|---|-----------------------------------------------------------------------------------|----------------------------|---|
| 1 |  | armrest   | 2 | 2 |  | With<br>wheel foot<br>tube | 2 |
| 3 |  | Foot tube | 2 | 4 |  | Main<br>frame              | 1 |

## INSTALLATION STEPS

1. Open the main frame: Unfold the left and right handrails of the main frame to both sides. After hearing a "ding" sound, the unfolding is completed, as shown in the right picture.

2. Install foot tubes: Insert the wheeled foot tubes into the two pipes in front of the main frame, so that the marbles can smoothly pop out of the marbles hole (note: each wheel is facing the outside of the main frame). Insert the foot tubes with sled foot pads into the two tubes behind the main frame, allowing the marbles to smoothly pop out of the marbles holes.

3. Install the handlebar: Insert the tube of the handlebar into the opening at the top of the main frame, so that the marble can smoothly pop out of the marble hole. Insert the handle tube into the tube under the hand rest, and the distance between the grip and the hand rest can be adjusted by the user. Then, lock the butterfly knob under the hand rest.

Nice Service: 100% tested before leaving the factory and 24-hour friendly email customer service.

**Manufacturer:** Zhongshan Bliss Medical Instrument Co., Ltd

**Address:** 3-6F & 7F-1, No. 63, Tongxing East Road, Yumin Community  
Xiaolan, Zhongshan, Guangdong, 528414, CHINA .

**Imported to AUS:** SIHAO PTY LTD. 1 ROKEVA STREET EASTWOOD NSW 2122  
Australia

**Imported to USA:** Sanven Technology Ltd. Suite 250, 9166 Anaheim Place,  
Rancho Cucamonga, CA 91730

|           |            |
|-----------|------------|
| <b>UK</b> | <b>REP</b> |
|-----------|------------|

YH CONSULTING LIMITED.

C/O YH Consulting Limited Office 147,  
Centurion House, London Road,  
Staines-upon-Thames, Surrey, TW18 4AX

|           |            |
|-----------|------------|
| <b>EC</b> | <b>REP</b> |
|-----------|------------|

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Mainzer Landstr.69, 60329 Frankfurt am Main.

# **VEVOR<sup>®</sup>**

**TOUGH TOOLS, HALF PRICE**

**Technical Support and E-Warranty Certificate**

**[www.vevor.com/support](http://www.vevor.com/support)**

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**Tillverkare:** Zhongshan Bliss Medical Instrument Co., Ltd
**Importerad till AUS:** SIHAO PTY LTD. 1 ROKEVA STREETEASTWOOD NSW 2122
**Importerad till USA:** Sanven Technology Ltd. Suite 250, 9166 Anaheim Place, Rancho
E-CrossStu GmbH