# **VEVOR<sup>®</sup>**

## **TOUGH TOOLS, HALF PRICE**

Technical Support and E-Warranty Certificate [www.vevor.com/support](http://www.vevor.com/support)

## **RAISED TOILET SEAT**

**Model: FLB760602**

We continue to be committed to provide you tools with competitive price.

"Save Half", "Half Price" or any other similar expressions used by us only represents an estimate of savings you might benefit from buying certain tools with us compared to the major top brands and does not necessarily mean to cover all categories of tools offered by us. You are kindly reminded to verify carefully when you are placing an order with us if you are actually saving half in comparison with the top major brands.

# VEVOR®

TOUGH TOOLS, HALF PRICE

## RAISED TOILET SEAT

**MODEL:** FBL760602

### NEED HELP? CONTACT US!

Have product questions? Need technical support? Please feel free to contact us:

**Technical Support and E-Warranty Certificate**  
**[www.vevor.com/support](http://www.vevor.com/support)**

This is the original instruction, please read all manual instructions carefully before operating. VEVOR reserves a clear interpretation of our user manual. The appearance of the product shall be subject to the product you received. Please forgive us that we won't inform you again if there are any technology or software updates on our product.

Warning-To reduce the risk of injury, users must read the instructions manual carefully.

## **WARNING:**

Read the manual completely to avoid harm, product damage, or warranty avoidance. Seek professional help if you encounter discomfort, pain or worsening symptoms. If necessary, use supervision in order to assemble and use this product correctly and safely. Do not exceed the 350 lbs weight limit.

Regularly check for secure assembly and contact support if in doubt. Ensure the product's legs are even and locked before use, and discontinue usage if there are signs of wear, damage, or missing parts. The product is to be used on flat, stable surfaces. Do not disassemble if not necessary, as it might compromise the longevity of the product. Do not modify the product unless specified in the manual. Use only parts approved by VEVOR for this specific product. Confirm the compatibility of the toilet rail with your toilet before first use. The product isn't built to withstand pressure or impact, and isn't to be used as a ladder or loaded with heavy objects.

Keep away from children and pets. Contains potentially hazardous small parts - choking hazard. Exercise caution during operation and assembly. Avoid extreme temperatures or open flames. If an allergic reaction occurs, cease use immediately and consult a healthcare professional.

## **Warranty Policy**

We warrant your new VEVOR product to be free from defects in materials and workmanship 3 years of the original purchase. However, please keep in mind that our guarantee policy does not cover counterfeit or replica items, nor does it extend to products purchased from unauthorized resellers or in used condition.

Please note that each purchase is eligible for only one guarantee redemption, and we cannot repeatedly refund, replace, or exchange your product. We also kindly ask that you handle the product with care and use it according to the instructions provided. Unfortunately, we cannot cover damage resulting from misuse or abuse of the product. This warranty is not transferable and applies only to the original purchaser.

## IMPORTANT

1. Don't exceed the weight capacity - 350 lbs max.
2. Use on stable surfaces only.
3. Regularly inspect for signs of wear, damage, or loose parts.
4. Clean regularly using a mild detergent and warm water, rinse off any soap residue. After cleaning, dry thoroughly.
5. Before use, confirm with your healthcare provider that the product meets your specific needs. The following guidelines are not a substitute for medical advice.

## SAVE THESE INSTRUCTION

### PARAMETER LIST

Model	FBL760602
Rated load (lbs)	350
Elevated height (in)	5
Product size(mm)	430*550*335

### PART LIST

When unpacking this product, check to make sure the parts listed below are included and carefully inspect for any damage that may have occurred during transit. If any part is missing or damaged, do not attempt to assemble or use the product.

NO	PICTURE	PART NAME	QTY	NO	PICTURE	PART NAME	QTY
1		Toilet seat	1	2		Left handrail	1
3		Right handrail	1				

## OPERATION

### 1. Installation Procedures


1. As shown in the picture: the left handrail tube aligned with the mouth of the tube into the marble pop-up, the armrest support two-position adjustment

2. As shown in the picture: install the right handrail according to the first step

3. After assembling the product and placing it on the toilet, adjust the knob so that the toilet seat is firmly stuck to the toilet.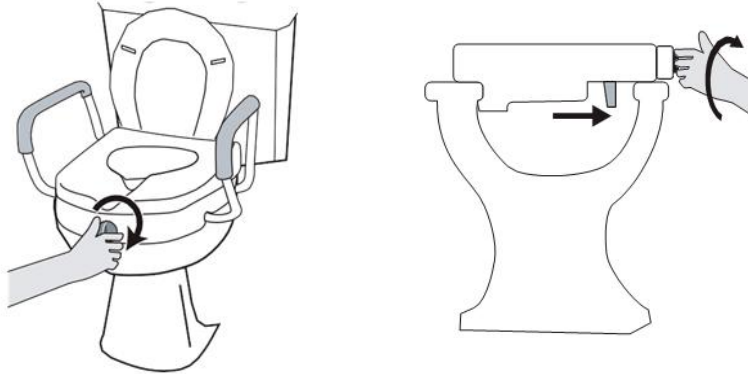

**Manufacturer:** Zhongshan Bliss Medical Instrument Co., Ltd

**Address:** 3-6F & 7F-1, No. 63, Tongxing East Road, Yumin Community Xiaolan, Zhongshan, Guangdong, 528414, CHINA

**Imported to AUS:** SIHAO PTY LTD. 1 ROKEVA STREET EASTWOOD NSW 2122 Australia

**Imported to USA:** Sanven Technology Ltd. Suite 250, 9166 Anaheim Place, Rancho Cucamonga, CA 91730

UK	REP
----	-----

YH CONSULTING LIMITED.

C/O YH Consulting Limited Office 147, Centurion House, London Road, Staines-upon-Thames, Surrey, TW18 4AX

EC	REP
----	-----

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**VEVOR<sup>®</sup>**

**TOUGH TOOLS, HALF PRICE**

**Technical Support and E-Warranty Certificate**

**[www.vevor.com/support](http://www.vevor.com/support)**

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**Tillverkare:** Zhongshan Bliss Medical Instrument Co., Ltd
**Importerad till AUS:** SIHAO PTY LTD. 1 ROKEVA STREET EASTWOOD NSW 2122 Australien
**Importerad till USA:** Sanven Technology Ltd. Suite 250, 9166 Anaheim Place, Rancho Cucamonga, CA 91730
E-CrossStu GmbH