VEVOR

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Fabrikant: FOSHAN SUNCARE MEDICAL PRODUCTS CO ., LTD . Adres:
2-3e verdieping van gebouw 65, Zhongnan Industry Garden, No.88 Haitian
Road, Gaoming District, Foshan City, Guangdong, 528500, CHINA .

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VEVOR

Affordable. Reliable. Home Improvement.

Raised Toilet Seat

Model: SC7060NA-L-4,SC7060Q-L-4,SC7060Q-S-4

VEVOR

Affordable. Reliable. Home Improvement.

Raised Toilet Seat

Model: SC7060NA-L-4, SC7060Q-L-4, SC7060Q-S-4

SC7060NA-L-4

SC7060Q-L-4

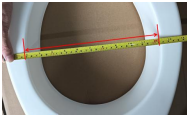
SC7060Q-S-4

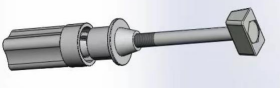
This is the original instruction, please read all manual instructions carefully before operating. VEVOR reserves a clear interpretation of our user manual. The appearance of the product shall be subject to the product you received. Please forgive us that we won't inform you again if there are any technology or software updates on our product.

Thank you very much for choosing this product, it should always be handled with care. Please read all of the instructions before using it. The information will help you achieve the best possible results.

SPECIFICATION

Model	SC7060NA-L-4	SC7060Q-L-4	SC7060Q-S-4	Note
Raised height	3.5 in	3.5 in	3.5 in	/
load	300lbs/ 136kg	300lbs/ 136kg	300lbs/ 136kg	/
Inner length(mm)	325	355	285	
Inner Width(mm)	250	265	247	
Mounting holes pitch(mm)	140	140	140	
Adapt to toilet size	Inner width \geq 250mm Inner length \geq 325mm	Inner width \geq 265mm, Inner length \geq 355mm	Inner width \geq 247mm, Inner length \geq 285mm	Please check the size of your toilet before choosing the raised toilet seat.

PACKAGE LIST

No.	Name	Picture	SC70 60NA -L-4	SC70 60Q- L-4	SC70 60Q- S-4
1	Raised Toilet Seat		1	1	1
2	Screw group		2	2	2
3	Armrest		2	0	0
4	Manual		1	1	1

CAUTIONS

1. Always be sure that the seat is correctly and securely locked in place before using.
2. Be sure that the locking of the seat is tight at all times, thus securing the seat to the toilet bowl. Physically challenged users should be supervised or assisted whenever using this product.
3. The raised toilet seat must not be used to support more than 300 lbs. DO NOT use the product overloaded. User's weight should be centered over toilet seat, not too forward or too far to one side, as unit may tip.
4. This product is designed for use with the most common types of commodes. It will not fit all commode types; be sure that the unit is

correctly installed, fits properly and is stable before use.

5. For models with armrests: The arms are for the purpose of providing assistance when getting on and off the seat. DO NOT try to use the arms to support full body weight during transfer, as this may result in failure of the unit and risk of injury.

6. If you have a style of commode that this product will not fit properly, or cannot be tightened securely as noted above, it is recommended that you discontinue usage.

7. Regularly check for secure assembly and contact support if in doubt. Ensure the product's legs are even and locked before use, and discontinue usage if there are signs of wear, damage, or missing parts. Do not disassemble if not necessary, as it might compromise the longevity of the product. Do not modify the product unless specified in the manual.

8. Exercise caution during operation and assembly. Avoid extreme temperatures or open flames. If an allergic reaction occurs, cease use immediately and consult a healthcare professional.

9. Before use, confirm with your healthcare provider that the product meets your specific needs. The following guidelines are not a substitute for medical advice.


INSTRUCTIONS

- **To install the armrests(Neglect this step for the products without armrests)**

1. Place each armrest into the holes in the bottom of the toilet seat. The metal buttons should be facing out.


2. Slide the armrest tube into position until the button clicks into the hole and a "click" sounds.

3. Repeat the process for the rest of the holes.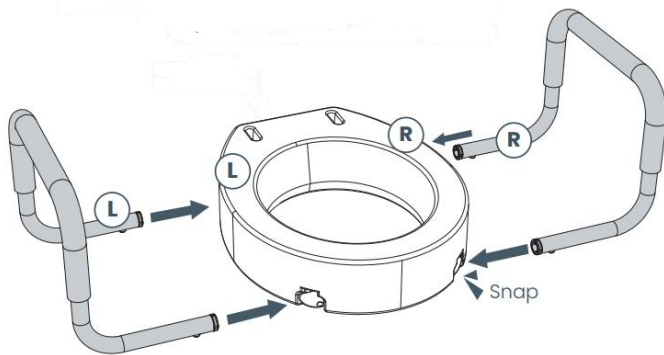

● **To install the Raised Toilet Seat**

1. Remove the existing toilet seat and lid from the toilet frame by unscrewing the seat screws located at the rear of the toilet seat.

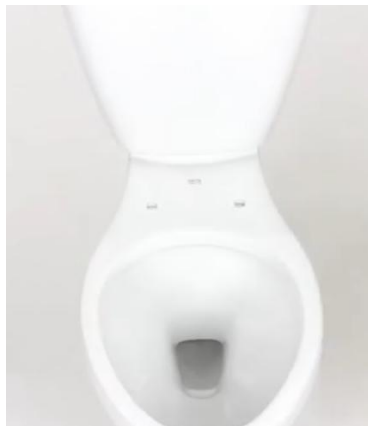2. Place the seat riser on the toilet frame and align the mounting holes.
3. Place the plastic inserts on the seat riser and align mounting holes with toilet frame, as below. Insert the mounting screws through the toilet seat.

4. Using the hand nut tighten the mounting screws. Do not fully tighten.
5. Be sure the toilet seat is straight and sits flush on the seat riser, then fully tighten the hand

MAINTENANCE

1. REGULAR CLEANING IS RECOMMENDED, USE A NON-ABRASIVE CLEANER OR DETERGENT. RINSE WITH WARM WATER AND DRY COMPLETELY. ABRASIVE CLOTHS MAY DAMAGE THE SURFACE.
2. PLEASE REGULARLY INSPECT YOUR RAISED TOILET SEAT WITH ARMS AND ITS PARTS FOR ANY CORROSION, BREAKAGE, COMPRESSION OR WEAR.

Manufacturer: FOSHAN SUNCARE MEDICAL PRODUCTS CO., LTD.
Address: 2-3 Floor of 65 Building, Zhongnan Industry Garden, No.88
Haitian Road, Gaoming District, Foshan City, Guangdong, 528500, CHINA.

Imported to AUS: SIHAO PTY LTD. 1 ROKEVA STREETEASTWOOD
NSW 2122 Australia

Imported to USA: Sanven Technology Ltd. Suite 250, 9166 Anaheim
Place, Rancho Cucamonga, CA 91730

UK	REP
-----------	------------

YH CONSULTING LIMITED. C/O YH Consulting
Limited Office 147, Centurion House, London
Road, Staines-upon-Thames, Surrey, TW18 4AX

EC	REP
-----------	------------

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E-CrossStu GmbH