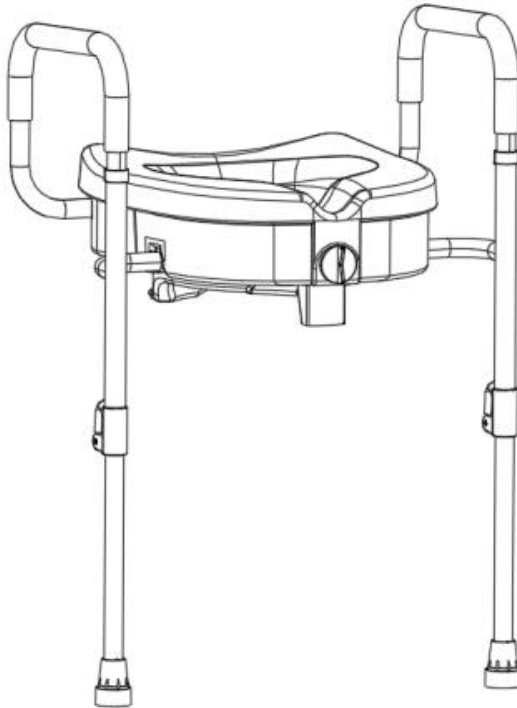
VEVOR

Affordable. Reliable. Home Improvement.

Raised Toilet Seat

Model: FBL760657

Model: FBL760657

This is the original instruction, please read all manual instructions carefully before operating. VEVOR reserves a clear interpretation of our user manual. The appearance of the product shall be subject to the product you received. Please forgive us that we won't inform you again if there are any technology or software updates on our product.

Thank you very much for choosing this product, it should always be handled with care. Please read all of the instructions before using it. The information will help you achieve the best possible results.

SPECIFICATION

Model	load	Seat lifting height	Height from armrest to floor
FBL760657	350lbs/159kg	5in(127mm)	670-820mm

PACKAGE LIST

No.	Name	Picture	QTY
1	Left armrest tube		1
2	Right armrest tube		1
3	Left armrest connecting pipe		1
4	Right armrest connecting pipe		1
5	Foot tube		2
6	Toilet raiser		1
7	Manual		1

CAUTIONS

1. Always be sure that the seat is correctly and securely locked in place before using.
2. Be sure that the locking of the seat is tight at all times. Physically challenged users should be supervised or assisted whenever using this product.
3. The raised toilet seat must not be used to support more than 300 lbs. DO NOT use the product overloaded. User's weight should be centered over toilet seat, not too forward or too far to one side, as unit may tip.
4. For models with armrests: The arms are for the purpose of providing assistance when getting on and off the seat. DO NOT try to use the arms to support full body weight during transfer, as this may result in failure of the unit and risk of injury.
5. Inspect all parts for shipping damage before assembly. If there is shipping damage - DO NOT USE. Contact dealer/carrier for further instructions.
6. Make certain that the height adjusting snap buttons fully protrude through the same respective hole of each leg extension. This will ensure that the leg extensions are securely locked in position and an even height is achieved.
7. Ensure ALL thumb screws are tightened.
8. Check the rubber tips on the leg extensions for rips, wear or if they are missing. Immediately replace any or all if any of these imperfections exist.
9. All legs extensions with rubber tips MUST touch the floor simultaneously at all times.
10. DO NOT stand on product.
11. Regularly check for secure assembly and contact support if in doubt. Ensure the product's legs are even and locked before use, and discontinue usage if there are signs of wear, damage, or missing parts. The product is to be used on flat, stable surfaces. Do not disassemble if not necessary, as it might compromise the longevity of the product. Do not modify the product unless specified in the manual.

12.Exercise caution during operation and assembly. Avoid extreme temperatures or open flames. If an allergic reaction occurs, cease use immediately and consult a healthcare professional.

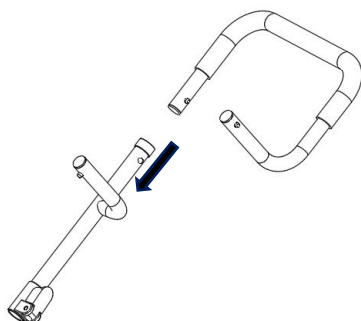
13.Clean regularly using a mild detergent and warm water, rinse off any soap residue. After cleaning, dry thoroughly.

14.Before use, confirm with your healthcare provider that the product meets your specific needs. The following guidelines are not a substitute for medical advice.

15.This product is designed for use with the most common types of commodes. It will not fit all commode types; be sure that the unit is correctly installed, fits properly and is stable before use.

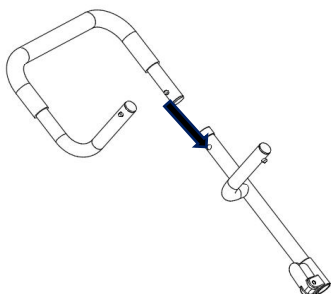
INSTRUCTIONS

1

Install the ①left armrest tube into the ③left armrest connecting tube.

2

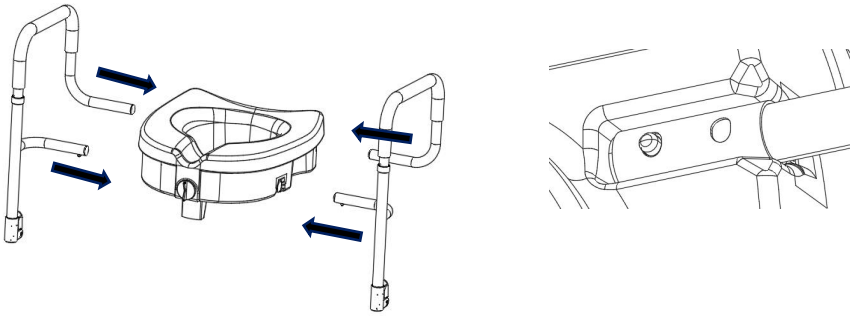
Install the ②right armrest tube into the ④right armrest connecting tube.

3

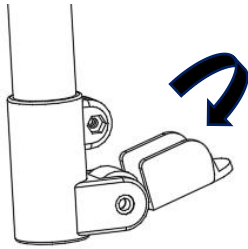
Align the two ends of the installed left and right armrest connecting pipes and insert them into the ⑥ toilet raiser. The armrest pipes can be adjusted in two

levels.

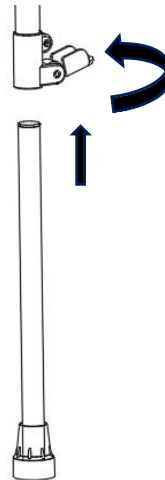
4

Insert the two ⑤ foot tubes into the tube clamp, close them tightly, and adjust the height of the foot tubes in conjunction with the toilet.

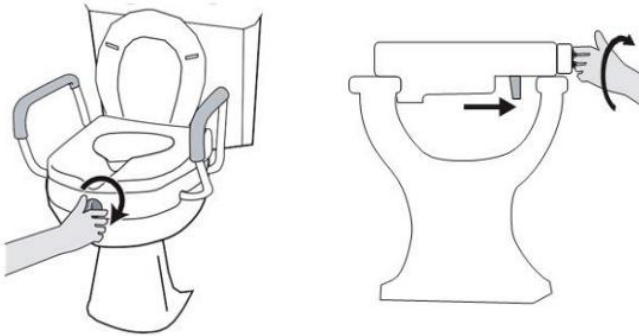
Open the pipe clamp

5

After the product is assembled and placed on the toilet, adjust the knob to make its toilet seat firmly stick to the toilet.

MAINTENANCE

1. REGULAR CLEANING IS RECOMMENDED, USE A NON-ABRASIVE CLEANER OR DETERGENT. RINSE WITH WARM WATER AND DRY COMPLETELY. ABRASIVE CLOTHS MAY DAMAGE THE SURFACE.
2. PLEASE REGULARLY INSPECT YOUR RAISED TOILET SEAT WITH ARMS AND ITS PARTS FOR ANY CORROSION, BREAKAGE, COMPRESSION OR WEAR.
3. To maintain the beauty of the original finish, clean with mild soap and rinse with warm water. Wipe bath seat dry after each use.

Manufacturer: Zhongshan Bliss Medical Instrument Co., Ltd

Address: 2F-2, 3-6F & 7F-1, No. 63, Tongxing East Road, Yumin Community, Xiaolan, Zhongshan, Guangdong, 528414, CHINA.

Imported to AUS: SIHAO PTY LTD. 1 ROKEVA STREET EASTWOOD NSW 2122 Australia

Imported to USA: Sanven Technology Ltd. Suite 250, 9166 Anaheim Place, Rancho Cucamonga, CA 91730

UK	REP
-----------	------------

YH CONSULTING LIMITED. C/O YH Consulting Limited Office 147, Centurion House, London Road, Staines-upon-Thames, Surrey, TW18 4AX

EC	REP
-----------	------------

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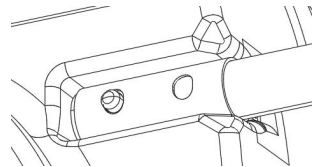
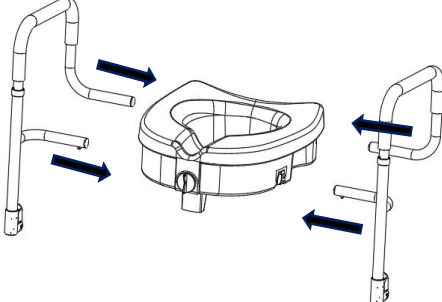
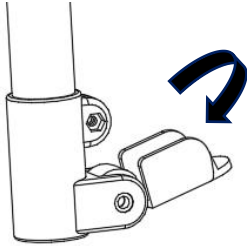
④ .

3

Align the two ends of the installed left and right armrest connecting pipes and
insert them into the ⑥ toilet raiser. The armrest pipes can be adjusted in two
levels.

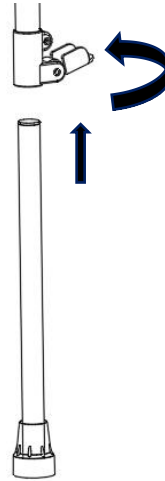
4

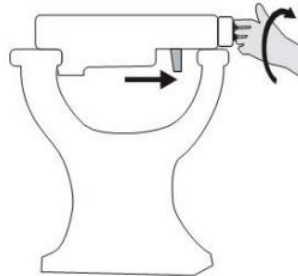
Open the pipe clamp

Insert the two ⑤foot tubes into the tube clamp, close them tightly, and adjust the height of the foot tubes in conjunction with the toilet.

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Hersteller: Zhongshan Bliss Medical Instrument Co., Ltd

Adresse: 2F-2, 3-6F & 7F-1, Nr. 63, Tongxing East Road, Yumin Community, Xiaolan, Zhongshan, Guangdong, 528414, CHINA.

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YH CONSULTING LIMITED. C/O YH Consulting
Limited Office 147, Centurion House, London
Road, Staines-upon-Thames, Surrey, TW18 4AX

EC	REP
-----------	------------

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Tillverkare: Zhongshan Bliss Medical Instrument Co., Ltd.
Importerad till Australien: SIHAO PTY LTD. 1 ROKEVA
Importerad till USA: Sanven Technology Ltd. Suite 250, 9166 Anaheim
E-CrossStu GmbH