## **Raised Toilet Seat**

**Model: XZY401**

**Model: XZY401**

This is the original instruction, please read all manual instructions carefully before operating. VEVOR reserves a clear interpretation of our user manual. The appearance of the product shall be subject to the product you received. Please forgive us that we won't inform you again if there are any technology or software updates on our product.

Thank you very much for choosing this product, it should always be handled with care. Please read all of the instructions before using it. The information will help you achieve the best possible results.

## SPECIFICATION

Model	load	Height from seat to floor	Height from armrest to floor
XZY401	300lbs/ 136kg	470-590mm	640-770mm

## PACKAGE LIST

No.	Name	Picture	QTY
A	Seat		1
B	Armrest		2
C	Reinforced crossbar		2
D	U-shaped tube		2
E	Leg connection tube		4
F	Leg with rubber tip		2
G	Leg with suction cup		2

H	M6 screw		4
I	Knob with screw		8
J	Hex key		1
K	Manual		1

## CAUTIONS

1. Always be sure that the seat is correctly and securely locked in place before using.
2. Be sure that the locking of the seat is tight at all times. Physically challenged users should be supervised or assisted whenever using this product.
3. The raised toilet seat must not be used to support more than 300 lbs. DO NOT use the product overloaded. User's weight should be centered over toilet seat, not too forward or too far to one side, as unit may tip.
4. For models with armrests: The arms are for the purpose of providing assistance when getting on and off the seat. DO NOT try to use the arms to support full body weight during transfer, as this may result in failure of the unit and risk of injury.
5. Inspect all parts for shipping damage before assembly. If there is shipping damage - DO NOT USE. Contact dealer/carrier for further

instructions.

6. Make certain that the height adjusting snap buttons fully protrude through the same respective hole of each leg extension. This will ensure that the leg extensions are securely locked in position and an even height is achieved.

7. Ensure ALL thumb screws are tightened.

8. Check the rubber tips on the leg extensions for rips, wear or if they are missing. Immediately replace any or all if any of these imperfections exist.

9. All four leg extensions with rubber tips MUST touch the floor simultaneously at all times.

10. DO NOT stand on product.

11. Regularly check for secure assembly and contact support if in doubt. Ensure the product's legs are even and locked before use, and discontinue usage if there are signs of wear, damage, or missing parts. The product is to be used on flat, stable surfaces. Do not disassemble if not necessary, as it might compromise the longevity of the product. Do not modify the product unless specified in the manual.

12. Exercise caution during operation and assembly. Avoid extreme temperatures or open flames. If an allergic reaction occurs, cease use immediately and consult a healthcare professional.

13. Clean regularly using a mild detergent and warm water, rinse off any soap residue. After cleaning, dry thoroughly.

14. Before use, confirm with your healthcare provider that the product meets your specific needs. The following guidelines are not a substitute for medical advice.

15. This product is designed for use with the most common types of commodes. It will not fit all commode types; be sure that the unit is correctly installed, fits properly and is stable before use.

## **INSTRUCTIONS**


Step1: insert the Leg connection tube (Part #E) into the Armrest (Part #B)


Step 2: Use the M6 screw (Part #H) to secure the Reinforced crossbar (Part #C) to the leg connection tube.

Step 3: Use the U-shaped tubes (Part #D) to connect the armrest frames on both sides and fix them with Knob (Part #I)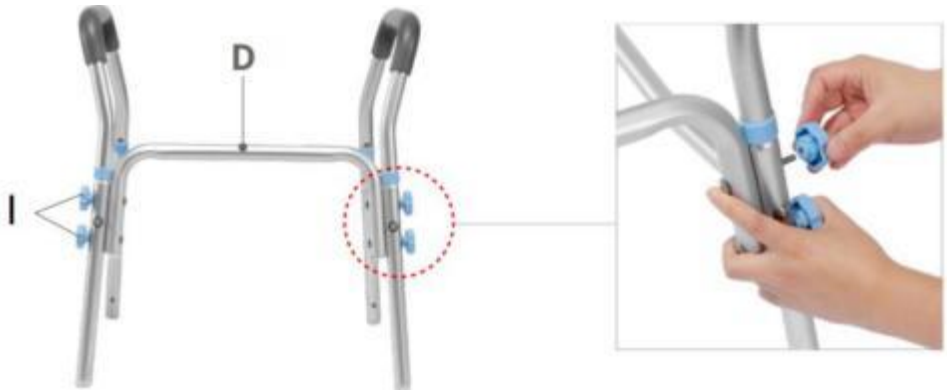

Step 4: Attach the Suction cup (Part #F) to the legs (Part #G), then insert the 4 legs into the connecting tubes as shown in the picture below

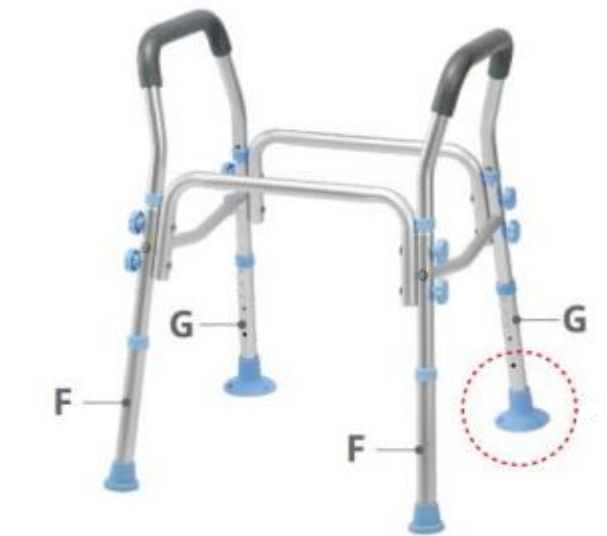
**Note:** Make sure all the Suction Cup legs are Lower 1 level than rubber tip leg.

**For example:** set the the rubber tip leg to 4, then you need to set the Suction Cup leg to 3.

Step 5: Finally secure the seat (Part #A) to the frame.

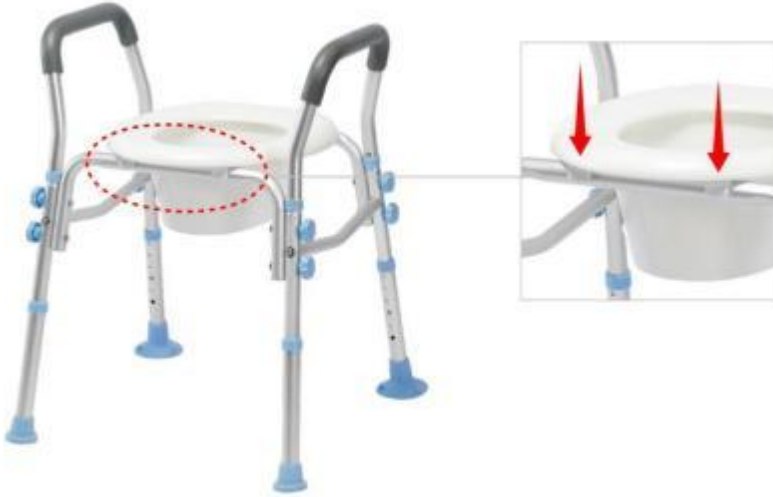## MAINTENANCE

1. REGULAR CLEANING IS RECOMMENDED, USE A NON-ABRASIVE CLEANER OR DETERGENT. RINSE WITH WARM WATER AND DRY COMPLETELY. ABRASIVE CLOTHS MAY DAMAGE THE SURFACE.
2. PLEASE REGULARLY INSPECT YOUR RAISED TOILET SEAT WITH ARMS AND ITS PARTS FOR ANY CORROSION, BREAKAGE, COMPRESSION OR WEAR.
3. To maintain the beauty of the original finish, clean with mild soap and rinse with warm water. Wipe bath seat dry after each use.

**Manufacturer:** HEALTH SOURCE MEDICAL EQUIPMENT (GUANGDONG) LTD.  
**Address:** 1st floor, No. 10 West Industrial Avenue, Haimai Industrial Zone, Gaohai Community, Danzao Town, Nanhai District, Foshan, Guangdong, 528223, CHINA.  
**Imported to AUS:** SIHAO PTY LTD. 1 ROKEVA STREETEASTWOOD

NSW 2122 Australia

**Imported to USA:** Sanven Technology Ltd. Suite 250, 9166 Anaheim Place, Rancho Cucamonga, CA 91730

<b>UK</b>	<b>REP</b>
-----------	------------

YH CONSULTING LIMITED. C/O YH Consulting Limited Office 147, Centurion House, London Road, Staines-upon-Thames, Surrey, TW18 4AX

<b>EC</b>	<b>REP</b>
-----------	------------

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**Tillverkare:** HÄLSOKÄLLA MEDICINSK UTRUSTNING (GUANGDONG) AB
Kina . **Importerad till Australien:** SIHAO PTY LTD. 1 ROKEVA
**Importerad till USA :** Sanven Teknologi Ltd. Suite 250, 9166
E-CrossStu GmbH