VEVOR[®]

TOUGH TOOLS, HALF PRICE

Technical Support and E-Warranty Certificate www.vevor.com/support

HUMAN BODY TEACHING MODELS

USER MANUAL

MODEL:LD-134,LD-135

We continue to be committed to provide you tools with competitive price.

"Save Half", "Half Price" or any other similar expressions used by us only represents an estimate of savings you might benefit from buying certain tools with us compared to the major top brands and does not necessarily mean to cover all categories of tools offered by us. You are kindly reminded to verify carefully when you are placing an order with us if you are actually saving half in comparison with the top major brands.

VEVOR®

TOUGH TOOLS, HALF PRICE

HUMAN BODY TEACHING
MODELS

MODEL:LD-134,LD-135

NEED HELP? CONTACT US!

Have product questions? Need technical support? Please feel free to contact us:

Technical Support and E-Warranty Certificate
www.vevor.com/support

This is the original instruction, please read all manual instructions carefully before operating. VEVOR reserves a clear interpretation of our user manual. The appearance of the product shall be subject to the product you received. Please forgive us that we won't inform you again if there are any technology or software updates on our product.

SAFETY INSTRUCTIONS

WARNING:

1. Read these instructions before using this product. Failure to do so can result in serious injury.
2. After use, please assemble all the parts and keep them vertically.
3. Avoid high temperature and direct sunlight.
4. Remove minor stains with neutral detergent. The use of alcohol, benzene and other substances may damage the product.

SAVE THESE INSTRUCTIONS

INSTRUCTIONS

- I. This model is suitable for nursing demonstration in health and nursing schools and medical colleges and universities at all levels.
- II. The model is all made of PVC material. The material is firm, durable and light to use.
- III. The joints of each part of the model can be moved, and the waist can be bent and all the components can be installed and removed.
- IV. Teaching and operating items of this model:**
(1) face washing and bed scrubbing; (2) oral nursing; (3) tracheotomy; (4) oxygen inhalation therapy (nasal obstruction and nasocatheterization); (5) nasal feeding; (6) gastric lavage; (7) intracardiac injection; (8) extrathoracic cardiac resuscitation; (9) Keywords pneumothorax; (10) thoracic puncture; (11) liver puncture; (12) kidney puncture; (13) celiac puncture; (14) bone marrow puncture; (15) lumbar puncture; (16) deltoid muscle injection; (17) hypodermic injection of deltoid muscle; (18) intravenous injection; (19) Venipuncture; (20) intravenous infusion; (21) intravenous transfusion; (22) female catheterization; (23) buttock muscle injection; (24) breast nursing.

V. operating instructions for the use of the model:

1.All parts of the body were injected,large and small needle pieces in the puncture site were made of soft plastic,and a small plastic bottle was packed after the small needle block,which could contain 20CC liquid,and the liquid of 20CC could be injected first as the pumping fluid before the drawing teaching.

2.The mouth and nose can be used as oxygen supply,nasal feeding and gastric lavage (the stomach is replaced by tube sac).

3.Intravenous injection,there is a large needle mass in the buttocks injection supply,nasal feeding and gastric lavage(the stomach is replaced by tube sac).

3.Intravenous injection,there is a large needle mass in the buttocks injection place,there is a skin tube(substitute vein tube),when the injection must be injected into the skin tube lumen,if a large amount of glucose infusion,the upper buttocks exposed skin tube should be put into the receptacle to loosen the clamp,And let the infusion run out.During the demonstration,a small amount of liquid is injected into the skin tube,and then the upper end of the skin tube can be clamped.

4.When injecting or puncturing,we must pay attention to the central part of the needle block.

5.The method of subcutaneous and intramuscular injection is the same as that of clinical operation.It can be injected with liquid,the sponge body can absorb water,and the water can be extruded and dried.Do not too hard in the operation of the needle,you can use the finger to gently press the needle block,and then dial the needle is good.

VI.During the emergency treatment of extra thoracic heart resuscitation,the hand can be pressed $1/3$ under the sternum to make the compressed place fall 2.5 cm.When the chest is heard whimpering,it will let go and restore,and the battery can be replaced when the battery is exhausted.o.If the injection or puncture needle block is damaged for a long time,it can be replaced.As long as the nut is rotated between the lower waist and the iron rod and the body is divided into two sections,the body can inspect the inner wall,spin down the small plastic bottle,and then remove the damaged

needle block to the outside and replace it with a new one. Then spin on a small plastic bottle, hip, arm needle block damage, can be pulled out, and then replaced with a new can. There are spare parts in each model of this kind of large and small needle block.

VII. Catheterization and enema teaching, can be inserted into the skin tube only for teaching (no liquid outflow).

VIII. Gastric lavage and nasal feeding can be used to insert nasal tubes from the nostril into the esophagus. Before insertion, the lower lip of the mouth (pressed inwards) can be opened and inserted with the finger, and then injected into the tube sac of the body. Hold the outer tube of the posterior buttocks and release the clamps when irrigation is done. (the skin of the back buttocks can be loosened by the release of the fluid.)

IX. After the practice is completed, check if the water in the tube bag is drained and clean it dry.

X. Use and maintenance of plastic models: This model is made of soft and rigid plastics.

1. In order to facilitate operation, the needle position can be slightly applied in the use of lubricating powder (that is, talc powder) to help smooth and smooth, with a clean test after use.

2. After the model is used, if there is dirt, you can use a little detergent to gently wipe away or a little soapy water and then clean with a cleaning cloth.

3. The model should be placed in a cool place, should not be exposed to the sun, and cover the cover.

PRODUCT PARAMETER

Model	LD-134	LD-135
Gender	Woman	Man
Material	PVC	PVC

PARTS LIST

Clothes	1	
Syringe 10ml	1	
Urinary Catheter	1	
Gastric Tube	1	
Injection Module	3	

Manufacturer: Shanghaimuxinmuyeyouxiangongsi
Address: Shuangchenglu 803nong11hao1602A-1609shi,
baoshanqu, shanghai 200000 CN.
Imported to AUS: SIHAO PTY LTD. 1 ROKEVA STREET-
EASTWOOD NSW 2122 Australia
Imported to USA: Sanven Technology Ltd. Suite 250, 9166
Anaheim Place, Rancho Cucamonga, CA 91730

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YH CONSULTING LIMITED.
C/O YH Consulting Limited Office 147, Centurion House,
London Road, Staines-upon-Thames, Surrey, TW18 4AX

VEVOR[®]

TOUGH TOOLS, HALF PRICE

Technical Support and E-Warranty Certificate

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E-CrossStu GmbH