install the backrest board④ with the backrest tube⑥ and secure it with the short bolt⑨ and washer⑧

SUBSEQUENT CONSULTATION: READ CAREFULLY

Warning:

- *Put the chair stably on a flat floor.
- *Stay away from open flames.
- *Building the chairs in good condition.
- *Avoid the overload at the chair.

Precautions before assembly:

- *Prepare a clean and dry mounting area.
- *Verify that the product is not damaged.
- *Check that none of the pieces are missing.

Maintenance:

- *Clean with a soft fabric.
- *Regularly remove leaves and / or water collected on the canvas.
- *Check the seating regularly.

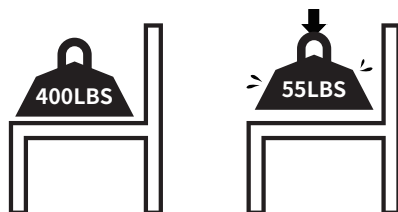
Storage conditions:

- *The fabric should be stored completely dry, in a clean and dry place to avoid the risks of mold.

Caution with the risks of pinching or cuts during assembly.

The manufacturer declines all responsibility in case of damages due to a bad installation.

Weight limited

VEVOR®

TOUGH TOOLS, HALF PRICE

Technical Support and E-Warranty Certificate www.vevor.com/support

BATH CHAIR

We continue to be committed to provide you tools with competitive price.

"Save Half", "Half Price" or any other similar expressions used by us only represents an estimate of savings you might benefit from buying certain tools with us compared to the major top brands and does not necessarily mean to cover all categories of tools offered by us. You are kindly reminded to verify carefully when you are placing an order with us if you are actually saving half in comparison with the top major brands.

VEVOR®

TOUGH TOOLS, HALF PRICE

Technical Support and E-Warranty Certificate

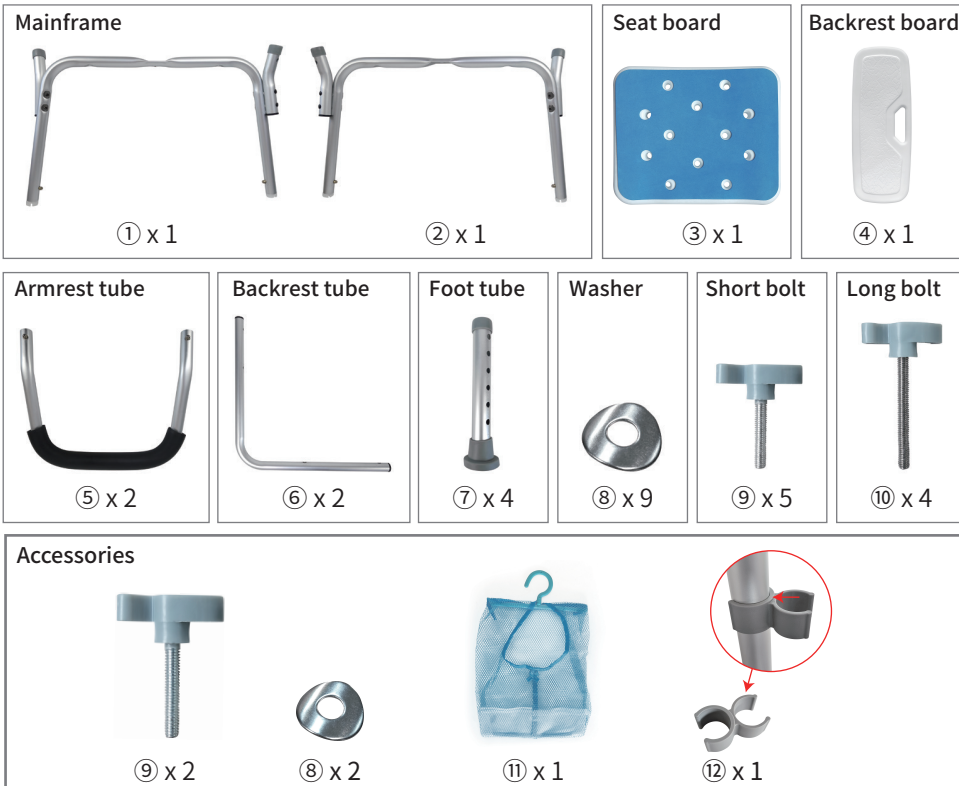
www.vevor.com/support

NEED HELP? CONTACT US!

Have product questions? Need technical support? Please feel free to contact us: **Technical Support and E-Warranty Certificate**
www.vevor.com/support

This is the original instruction, please read all manual instructions carefully before operating. VEVOR reserves a clear interpretation of our user manual. The appearance of the product shall be subject to the product you received. Please forgive us that we won't inform you again if there are any technology or software updates on our product.


Part List

Step1

① x 1 ② x 1 ⑤ x 2

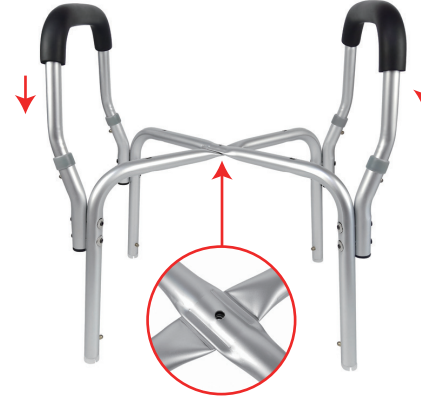Install the mainframe ①② with the armrest tube ⑤ till the marble is secured

Note: the direction of the pipe

Step2

② x 1 ⑨ x 1 ⑧ x 1

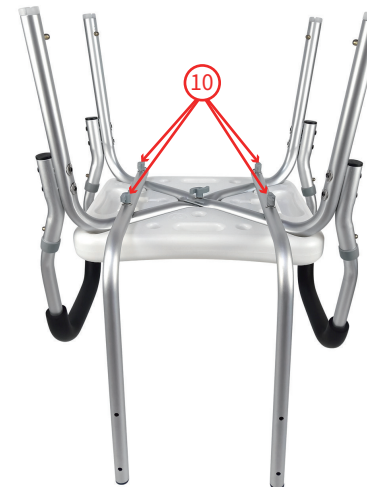
Install the mainframe with the seat board ② and secure it with the short bolt ⑨ and washer ⑧

Step3

⑥ x 2 ⑩ x 4 ⑧ x 4

install the backrest tube ⑥ with the mainframe and secure it with the long bolt ⑩ and washer ⑧


Step4

⑦ x 4

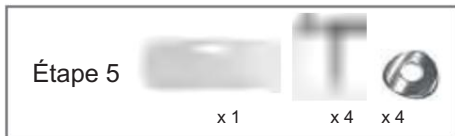install the foot tube ⑦ with the mainframe till the marble is secured and adjust the same height desired.

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