VEVOR

Affordable. Reliable. Home Improvement.

Raised Toilet Seat

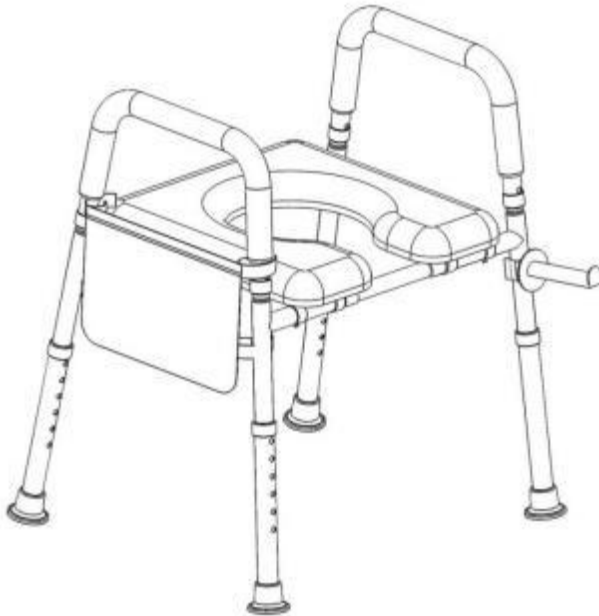
Model: XZY402

VEVOR

Affordable. Reliable. Home Improvement.

Raised Toilet Seat

Model: XZY402

This is the original instruction, please read all manual instructions carefully before operating. VEVOR reserves a clear interpretation of our user manual. The appearance of the product shall be subject to the product you received. Please forgive us that we won't inform you again if there are any technology or software updates on our product.

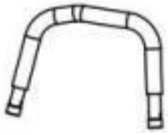
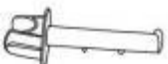
Thank you very much for choosing this product, it should always be handled with care. Please read all of the instructions before using it. The information will help you achieve the best possible results.

SPECIFICATION

Model	load	Height from seat to floor	Height from armrest to floor
XZY402	400lbs/ 181kg	450-570mm	650-770mm

PACKAGE LIST


No.	Name	Picture	QTY
A	Connecting tube		4
B	Armrest		2
C	Seat plate		1
D	Transverse tube		2
E	C-Clamp		4
F	Bag		1
G	Leg		4
H	Tissue hook		1


I	Manual		1
---	--------	--	---

INSTRUCTIONS

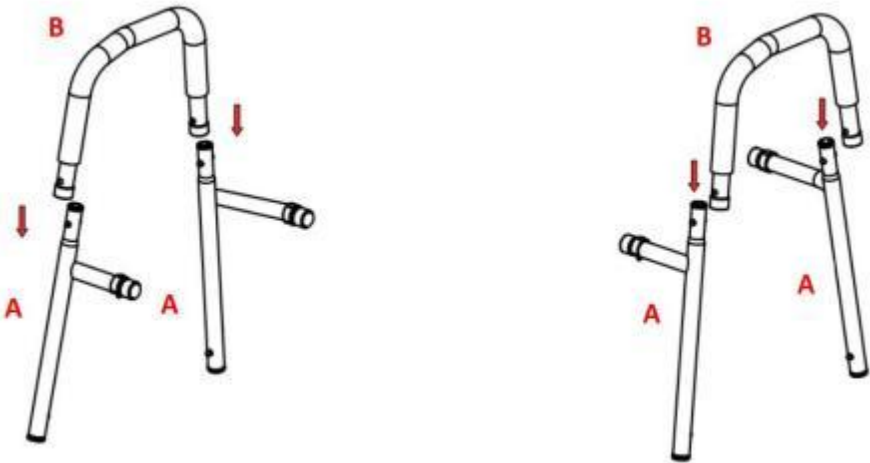
Step1: Install the armrest(Tab B) into the Connecting tube(Tab A). Make sure the marbles run smoothly.

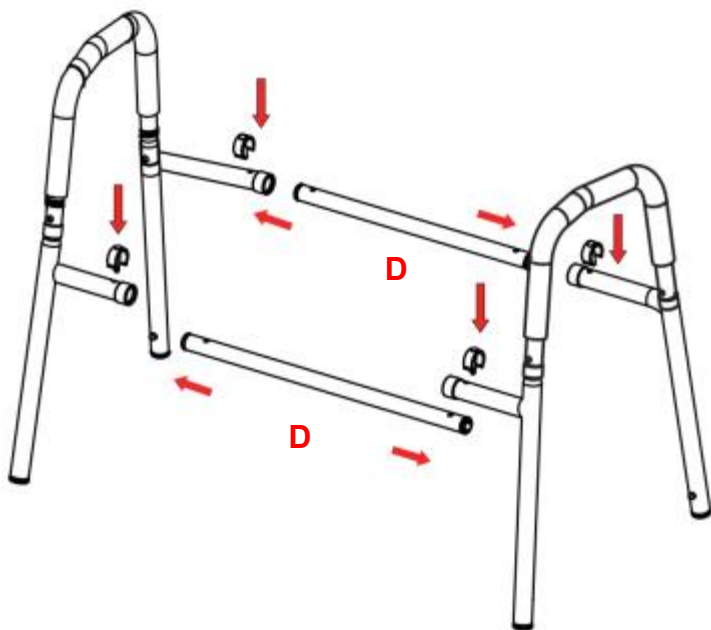Step2: Connect transverse tube(Tab D) and left/right tube group with C-clamp. Warning: check carefully the C-clamp is right to insert the hole otherwise, it may cause serious injury.

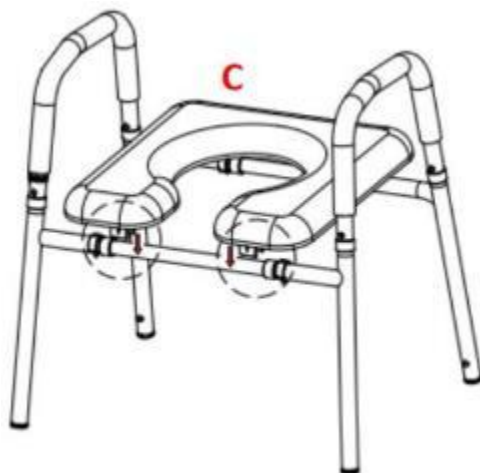
Step3: Install the seat plate(Tab C) to the transverse tube group.

Step4: Attach the cloth bag(Tab F) to the armrest.

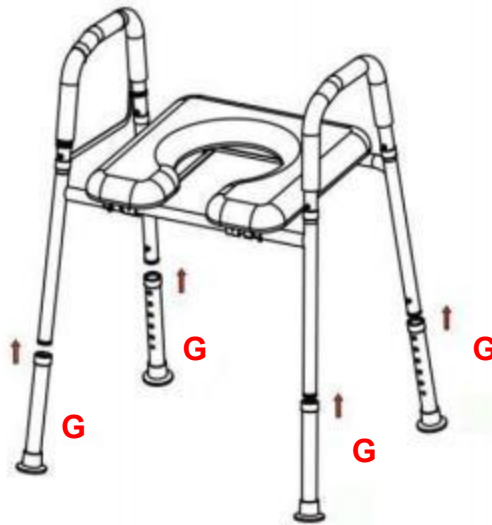Step5: Connect the foot tube(Tab G) to the main frame. Make sure the marbles smoothly.

Height adjustment

1. Before use, adjust the uniform height of the four legs according to your specific needs.
2. Locked in place by pushing up or pulling down on the foot tube.

CAUTIONS

1. Always be sure that the seat is correctly and securely locked in place before using.
2. Be sure that the locking of the seat is tight at all times. Physically challenged users should be supervised or assisted whenever using this product.
3. The raised toilet seat must not be used to support more than 300 lbs. DO NOT use the product overloaded. User's weight should be centered over toilet seat, not too forward or too far too one side, as unit may tip.
4. For models with armrests: The arms are for the purpose of providing assistance when getting on and off the seat. DO NOT try to use the arms to support full body weight during transfer, as this may result in failure of the unit and risk of injury.
5. Inspect all parts for shipping damage before assembly. If there is shipping damage - DO NOT USE. Contact dealer/carrier for further instructions.
6. Make certain that the height adjusting snap buttons fully protrude through the same respective hole of each leg extension. This will ensure that the leg extensions are securely locked in position and an even height is achieved.
7. Ensure ALL thumb screws are tightened.
8. Check the rubber tips on the leg extensions for rips, wear or if they are missing. Immediately replace any or all if any of these imperfections exist.
9. All four leg extensions with rubber tips MUST touch the floor simultaneously at all times.
10. DO NOT stand on product.
11. Regularly check for secure assembly and contact support if in doubt. Ensure the product's legs are even and locked before use, and discontinue usage if there are signs of wear, damage, or missing parts. The product is to be used on flat, stable surfaces. Do not disassemble if not necessary,

as it might compromise the longevity of the product. Do not modify the product unless specified in the manual.

12. Exercise caution during operation and assembly. Avoid extreme temperatures or open flames. If an allergic reaction occurs, cease use immediately and consult a healthcare professional.

13. Clean regularly using a mild detergent and warm water, rinse off any soap residue. After cleaning, dry thoroughly.

14. Before use, confirm with your healthcare provider that the product meets your specific needs. The following guidelines are not a substitute for medical advice.

15. This product is designed for use with the most common types of commodes. It will not fit all commode types; be sure that the unit is correctly installed, fits properly and is stable before use.

MAINTENANCE

1. REGULAR CLEANING IS RECOMMENDED, USE A NON-ABRASIVE CLEANER OR DETERGENT. RINSE WITH WARM WATER AND DRY COMPLETELY. ABRASIVE CLOTHS MAY DAMAGE THE SURFACE.

2. PLEASE REGULARLY INSPECT YOUR RAISED TOILET SEAT WITH ARMS AND ITS PARTS FOR ANY CORROSION, BREAKAGE, COMPRESSION OR WEAR.

3. To maintain the beauty of the original finish, clean with mild soap and rinse with warm water. Wipe bath seat dry after each use.

Manufacturer: HEALTH SOURCE MEDICAL EQUIPMENT (GUANGDONG) LTD.

Address: 1st floor, No. 10 West Industrial Avenue, Haimai Industrial Zone, Gaohai Community, Danzao Town, Nanhai District, Foshan, Guangdong, 528223, CHINA.

Imported to AUS: SIHAO PTY LTD. 1 ROKEVA STREETEASTWOOD
NSW 2122 Australia

Imported to USA: Sanven Technology Ltd. Suite 250, 9166 Anaheim
Place, Rancho Cucamonga, CA 91730

UK	REP
-----------	------------

YH CONSULTING LIMITED. C/O YH Consulting
Limited Office 147, Centurion House, London
Road, Staines-upon-Thames, Surrey, TW18 4AX

EC	REP
-----------	------------

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Technical Support and E-Warranty Certificate
www.vevor.com/support

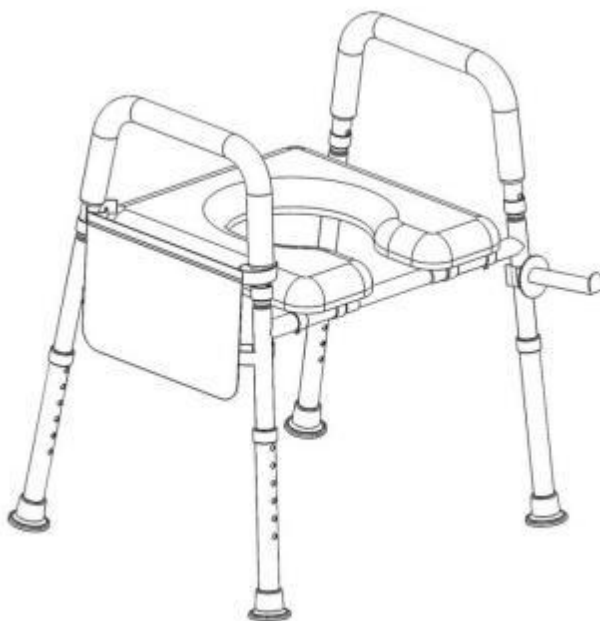
VEVOR

Affordable. Reliable. Home Improvement.

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Tillverkare: HÄLSOKÄLLA MEDICINSK UTRUSTNING (GUANGDONG) AB
Kina . **Importerad till Australien:** SIHAO PTY LTD. 1 ROKEV A
Importerad till USA: Sanven Technology Ltd. Suite 250 , 9166
E-CrossStu GmbH