VEVOR[®]

TOUGH TOOLS, HALF PRICE

Technical Support and E-Warranty Certificate www.vevor.com/support

WALKING CANE

We continue to be committed to provide you tools with competitive price. "Save Half", "Half Price" or any other similar expressions used by us only represents an estimate of savings you might benefit from buying certain tools with us compared to the major top brands and does not necessarily mean to cover all categories of tools offered by us. You are kindly reminded to verify carefully when you are placing an order with us if you are actually saving half in comparison with the top major brands.

VEVOR[®]

TOUGH TOOLS, HALF PRICE

WALKING CANE

MODEL: GB-8170

NEED HELP? CONTACT US!

Have product questions? Need technical support? Please feel free to contact us:

Technical Support and E-Warranty Certificate

www.vevor.com/support


This is the original instruction, please read all manual instructions carefully before operating. VEVOR reserves a clear interpretation of our user manual. The appearance of the product shall be subject to the product you received. Please forgive us that we won't inform you again if there are any technology or software updates on our product.

PARAMETER


Model	NO.	Product size	Product color
GB-8170	SKU3	130*100*770~945mm	Blue
GB-8170	SKU4	130*100*770~945mm	Purple
GB-8170	SKU5	130*100*770~945mm	Black

FDA Registration No.: 3012468755


SETTING UP YOUR WALKING CANE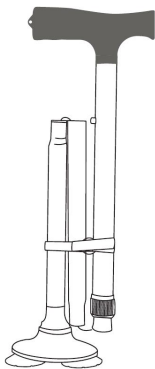


1. Remove from packaging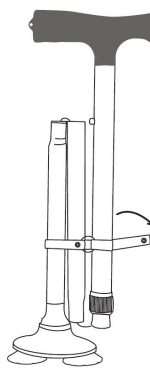


2. Unbutton strap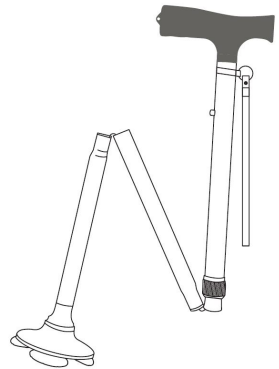


3. Unfold

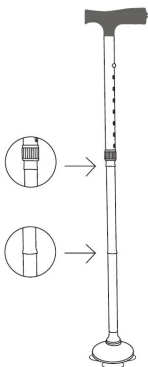
4. Important: Check to make sure that all joints are secure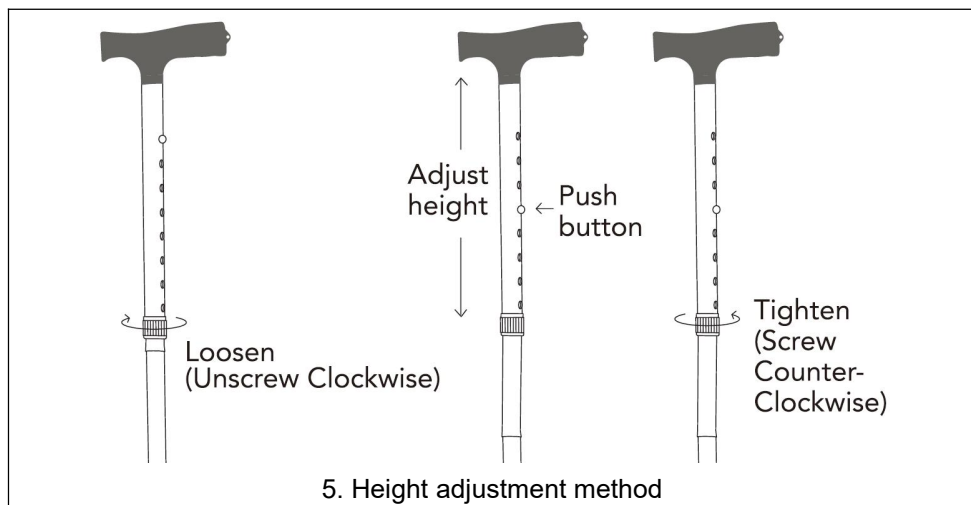


PUTTING AWAY YOUR WALKING CANE

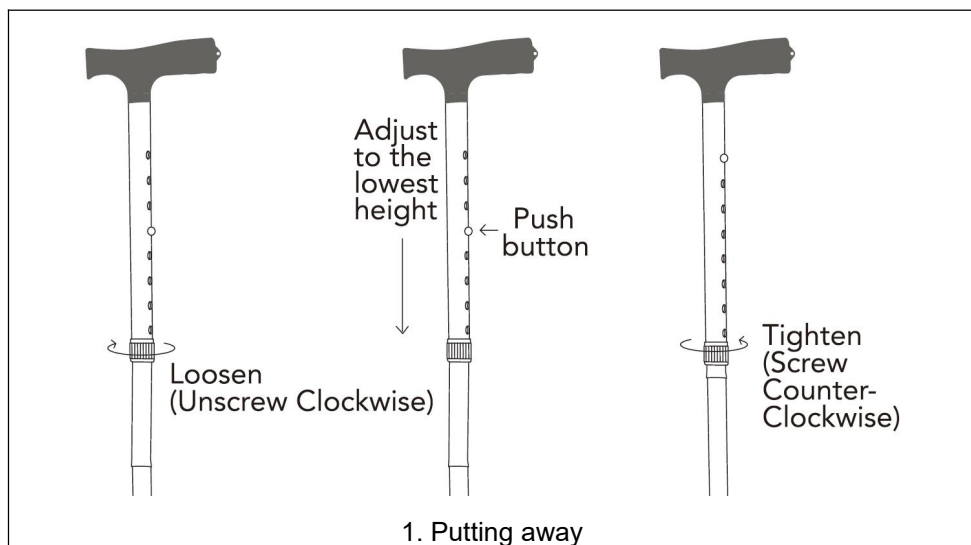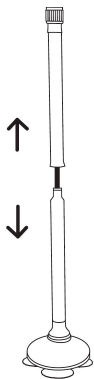

2. Pull apart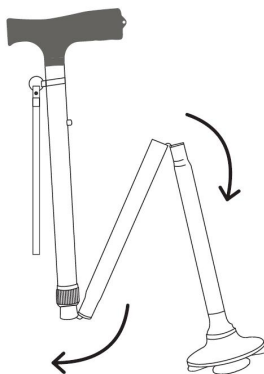

3. Fold up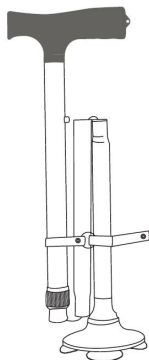

4. Button strap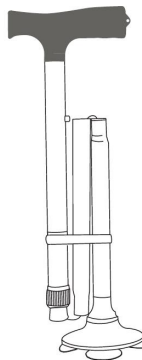


5. Ready for storage

OPTIMAL USE OF THE WALKING CANE HEIGHT OF THE WALKING CANE

The walking cane can adjust between 30.3 to 37.2 inches tall.

The best height to have your walking cane at is your hips.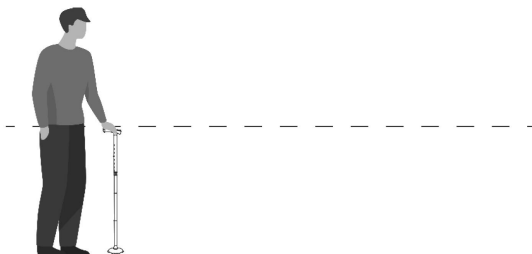

FEATURES

1. Foldable design for easy carrying and storage with Ergonomic handle
2. Base of cane consists of 'Pivot ball design' provides extra grip & consists of 4 TPR feet design with improved stability.
3. Eight adjustable heights 30.3 inches to 37.2 inches Perfect for people between 5 ft and 6 ft 6 inch.

MAINTENANCE

1. Does not use the cane if the cord is impaired; check the elasticity of cord before use every time.
2. Check if the screws of rubber feet are secure, if not use a screw driver to tighten the feet for proper use. Replace the rubber feet when they stop working properly.

LIMITATIONS

1. Walking cane is not intended to fully hold the weight of a person.
2. Walking cane is not intended to use by persons over 250pounds (113kg)
3. Walking cane is not meant to be used on slippery or wet surfaces.



SAFETY PRECAUTIONS

Please read the safety precautions before use in order to reduce the risk of injury:

1. Consult a physician before use of any walking aid.
2. Do not use the walking cane unless the segments and push button are secure and tightened to lock in place.
3. When straightening the segments keep your fingers away to avoid being pinched.
4. Do not stretch the internal elastic cord more than necessary
5. Do not use the walking cane if the elastic cord is cut or damaged
6. If the handle or bottom base becomes loose, don't use the walking cane.

WARNING

DO NOT disassemble the body of the cane. This will results the damage of elastic cord inside the cane, uncontrolled release of the elastic cord could cause severe injury.

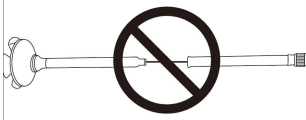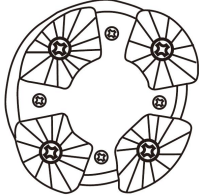Do not over pull the segments and stretch out the elastic cord. 	Be sure the screws are tight on the bottom of the cane. Make sure to replace the foot every 6 months. 	The walking cane is designed to adapt to many environments but make sure to not use your walking cane on surfaces like these:   Uneven floors Throw rugs
---	--	--

Manufacturer: NINGBO GAOBO SPORTS CO.,LTD.

Address: NO.1 XIAJIN, SHENZHEN, NINGHAI, Ningbo, Zhejiang, 315614, CHINA

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YH CONSULTING LIMITED.
C/O YH Consulting Limited Office 147, Centurion House,
London Road, Staines-upon-Thames, Surrey, TW18 4AX

VEVOR[®]

TOUGH TOOLS, HALF PRICE

Technical Support and E-Warranty Certificate

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Fabrikant: NINGBO GAOBO SPORTS CO.,LTD.
Geïmporteerd naar AUS: SIHAO PTY LTD. 1 ROKEVA STREETEASTWOOD NSW 2122 Australië
Geïmporteerd naar de VS: Sanven Technology Ltd. Suite 250, 9166 Anaheim Place, Rancho Cucamonga, CA 91730
E-CrossStu GmbH
C/O YH Consulting Limited Kantoor 147, Centurion House,