# **VEVOR<sup>®</sup>**

## **TOUGH TOOLS, HALF PRICE**

Technical Support and E-Warranty Certificate

[www.vevor.com/support](http://www.vevor.com/support)

### **ROLLING WALKER**

**MODEL:FBL427112**

We continue to be committed to provide you tools with competitive price. "Save Half", "Half Price" or any other similar expressions used by us only represents an estimate of savings you might benefit from buying certain tools with us compared to the major top brands and does not necessarily mean to cover all categories of tools offered by us. You are kindly reminded to verify carefully when you are placing an order with us if you are actually saving half in comparison with the top major brands.

# VEVOR®

TOUGH TOOLS, HALF PRICE

ROLLING WALKER

MODEL: FBL427112

*The product color is for reference only and is mainly promoted on the website*

## NEED HELP? CONTACT US!

Have product questions? Need technical support? Please feel free to contact us:

**Technical Support and E-Warranty Certificate**  
**[www.vevor.com/support](http://www.vevor.com/support)**

This is the original instruction, please read all manual instructions carefully before operating. VEVOR reserves a clear interpretation of our user manual. The appearance of the product shall be subject to the product you received. Please forgive us that we won't inform you again if there are any technology or software updates on our product.

# INSTRUCTIONS

## **Preface**

First of all, let us express our heartfelt thanks to our customers who have purchased our equipment. This manual is for customer service to make better use of this equipment. We hope the company's products and services to bring you the canal courtesy.

## **Pay attention**

This product must be used in strict accordance with the use method of this manual, misuse of any other methods caused by misuse. The Company does not give any guarantee of any kind, including (but not limited to) the marketability offered to the Company for a particular purpose. Implied warranties with fitness.

The company accepts no liability. The Company shall not be liable for any errors contained in this material, or for any accidental or consequential damages resulting from the provision, actual performance or use of this manual.

This manual contains proprietary information protected by patent law. All rights reserved. Without the written consent of the Company, this manual is not allowed to be photocopied, copied, or translated any part into another language.

The content contained in the manual may be changed without the user's notice.

## **Manufacturer's liability**

The Company shall be held responsible for the safety and reliability of the performance of the equipment only if the assembly operation, returning, improvement and maintenance are carried out by personnel approved by the Company, the electrical equipment concerned conforms to national standards, and the equipment is used in accordance with the operating instructions.

## **Guarantee**

Manufacturing process and raw materials: The Company guarantees that the Equipment (except parts) are in regular use and repair condition, free of

manufacturing process and raw materials faults for one year from the date of shipment. None of our inspections is perfect.

## **PRECAUTIONS**


### **Precautions for use**

#### **Warning:**

1. If the equipment feels loose, falls or is damaged, please contact the company in time;
2. Keep work area clean and well lit. Cluttered or dark areas invite accidents.
- 3.Keep children and bystanders away while operating, distractions will cause you to lose control.
- 4.During normal use, please use with the accompaniment of family or friends.
- 5.Reasonably use according to one's own situation, and do not overuse when the body is not in a timely manner.
- 6.BEFORE USE, PLEASE CONFIRM WHETHER THE COMPONENTS ARE COMPLETE, WHETHER THE BRAKE IS SENSITIVE, AND ASSEMBLE THE ENTIRE PRODUCT IN STRICT ACCORDANCE WITH THE INSTRUCTIONS.
- 7.CLEANING PRODUCTS NEED TO BE WASHED WITH WATER AND DRY IN TIME. IF THE SURFACE COATING OF THE PRODUCT HAS SCRATCHES AND FALLS OFF, REPAIR MATERIALS NEED TO BE USED IN TIME TO ACHIEVE THE BEST ANTI-RUST EFFECT.
- 8.PLEASE ENSURE THAT THE REAR WHEEL OF THE PRODUCT IS LOCKED TO AVOID SLIDING WHEN THE USER STAYS IN THE SLOPING ENVIRONMENT.
- 9.PLEASE ENSURE THAT THE FOLDING MECHANISM OF THE SEAT PLATE IS LOCKED WHEN USING THIS PRODUCT.

## **SAVE THESE INSTRUCTIONS**

## PRODUCT PARAMETERS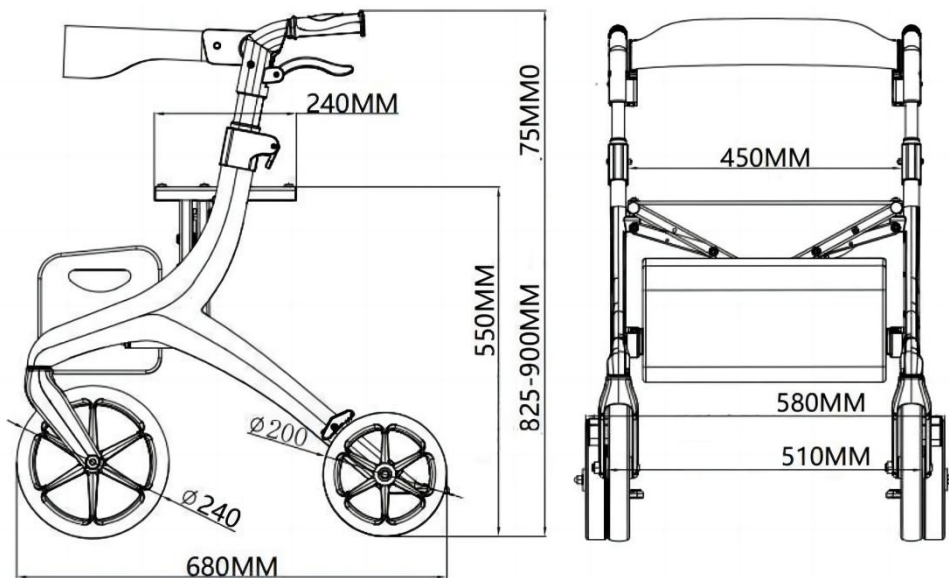


Product Name	Rolling Walker
model	FBL427112
colour	Blue/Black/Red
Wheel size	Front 9.5in/Back 8in
Load bearing capacity/lb	300
INNER SEAT WIDTH	450MM
SEAT WIDTH	240MM
HEIGHT OF THE SEAT PLATE FROM THE GROUND	550MM
ELEVATION RANGE	75MM

## INSTALLATION STEPS

THE REFERENCE INSTRUCTION MANUAL IS AS FOLLOW.  
(DIFFERENT MODLE'S OUT-LOOK IS NOT BE SAME, BUT THE USAGE METHOD ARE SAME.)

### 1. Handrail height adjustment instructions;

As shown in the figure, turn the wrench 90 degrees counterclockwise to unlock the upgrade mechanism. Adjust according to the adjustment scale to the required height and press clockwise back to the original position to lock.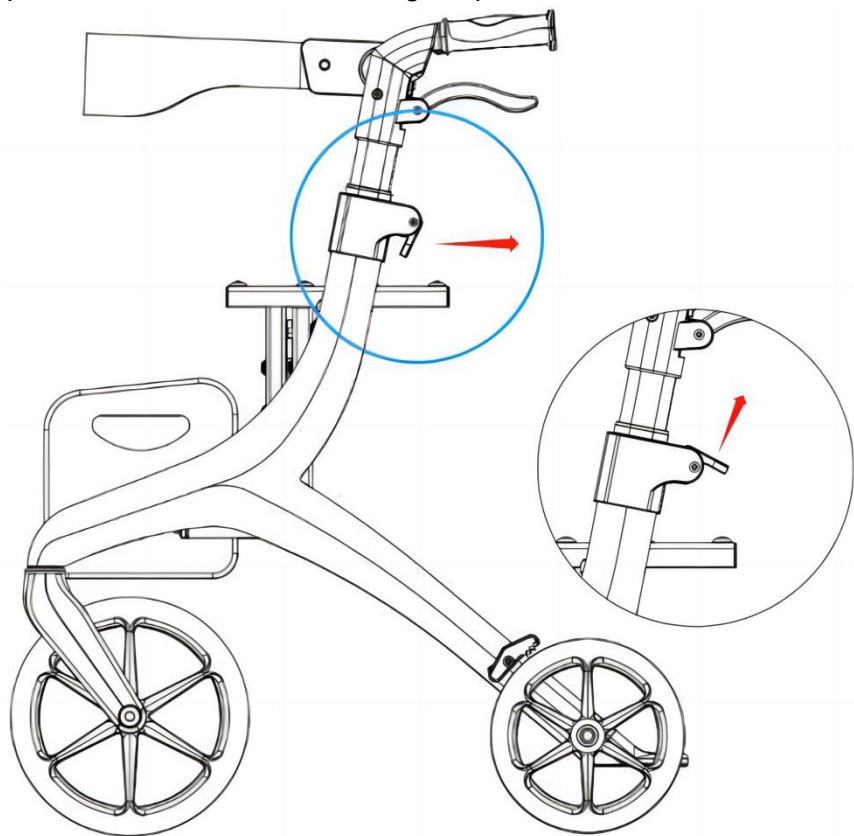


### 2. Parking function description;

As shown in the figure, turn the wrench clockwise down to realize the


parking lock

function, and turn counterclockwise upward to unlock;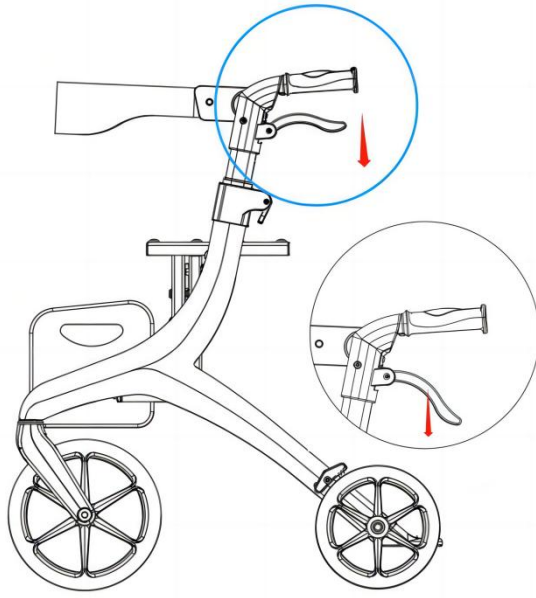

3. Closing function description;

As shown in the figure, pull the handle in the center of the seat cloth upward to realize the folding and retracting function;

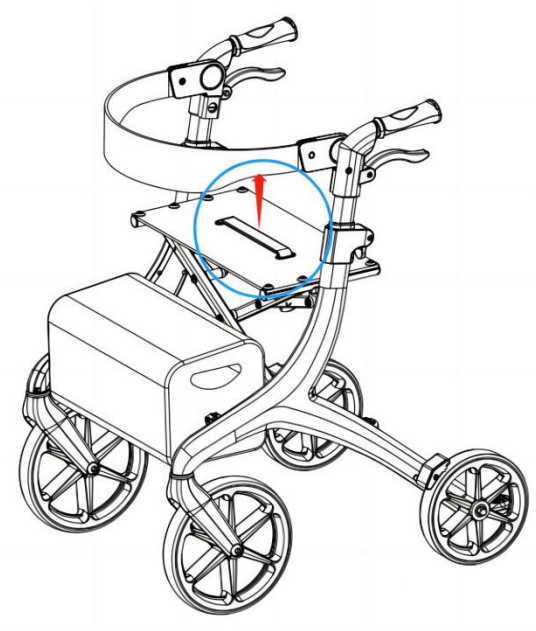**THIS PRODUCT IS A NEW GENERATION OF ROLLATOR DEVELOPED BY OUR COMPANY. ASSIST THE ELDERLY, LOWER LIMB WEAKNESS, MOBILITY AND OTHER PEOPLE WALKING, REST. IN ORDER TO ENSURE SAFETY AND AVOID PROBLEMS DURING USE, PLEASE BE SURE TO READ THIS MANUAL CAREFULLY BEFORE USE.**

**CHECK ALL FUNCTIONS IF ALL PARTS WORK WELL. PRESS THE REAR WHEEL BRAKE AND TRY TO PUSH IT TO CHECK WHETHER THE BRAKE SWITCH IS EFFECTIVE; UNLOCK THE REAR WHEEL BRAKE, PUSH THE PRODUCT TO MOVE, CHECK WHETHER THE MOVEMENT IS SMOOTH; UNLOCK THE LIFTING LIMIT SWITCH, ADJUST THE HEIGHT UP AND DOWN, AND TEST**

## **WHETHER THE HEIGHT ADJUSTMENT IS SMOOTH;**

### **About this item**

1. Stable & Not Easy to Tip Over: 8" front rear wheels, 360° swivel front wheels for easy maneuvering, sturdy enough for indoor and outdoor smooth ride.
2. Convenient to Go Outside: Zippered carrying pouch, keep all your important items instantly within reach, free your hands to fully enjoy your life.
3. Nice Service: 100% tested before leaving the factory and 24-hour friendly email customer service.
4. This product comes with self braking and auxiliary seat cushion functions. When the operator needs to rest, they can turn on the self braking and sit on the seat cushion to rest.
5. There is a portable storage bag under the product cushion, which is convenient for storing some miscellaneous items and purchasing some daily necessities for storage.

**Manufacturer:** Shanghaimuxinmuyeyouxiangongsi

**Address:** Baoshanqu Shuangchenglu 803long 11hao 1602A-1609shi Shanghai

**Imported to AUS:** SIHAO PTY LTD, 1 ROKEVA STREETEASTWOOD NSW 2122  
Australia

**Imported to USA:** Sanven Technology Ltd.

Suite 250,9166 Anaheim Place, Rancho Cucamonga, CA 91730

SHUNSHUN GmbH

Römeräcker 9 Z2021,76351

Linkenheim-Hochstetten, Germany

Pooledas Group Ltd

Unit 5 Albert Edward House, The

Pavilions Preston, United Kingdom

**VEVOR<sup>®</sup>**

**TOUGH TOOLS, HALF PRICE**

**Technical Support and E-Warranty Certificate**

**[www.vevor.com/support](http://www.vevor.com/support)**

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[www.vevor.com/support](http://www.vevor.com/support)

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**MODEL: FBL427112**

We continue to be committed to provide you tools with competitive price.

"Save Half", "Half Price" or any other similar expressions used by us only represents an estimate of savings you might benefit from buying certain tools with us compared to the major top brands and does not necessarily mean to cover all categories of tools offered by us. You are kindly reminded to verify carefully when you are placing an order with us if you are actually saving half in comparison with the top major brands.

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**Importato in AUS:** SIHAO PTY LTD, 1 ROKEVA STREETEASTWOOD NSW
**Importato negli Stati Uniti:** Sanven Technology Ltd.