VEVOR

Affordable. Reliable. Home Improvement.

Raised Toilet Seat

Model: SC7060A,SC7060B

VEVOR

Affordable. Reliable. Home Improvement.

Raised Toilet Seat

Model: SC7060A,SC7060B

SC7060A

SC7060B

This is the original instruction, please read all manual instructions carefully before operating. VEVOR reserves a clear interpretation of our user manual. The appearance of the product shall be subject to the product you received. Please forgive us that we won't inform you again if there are any technology or software updates on our product.

Thank you very much for choosing this product, it should always be handled with care. Please read all of the instructions before using it. The information will help you achieve the best possible results.

SPECIFICATION

Model	Raised height	load	Adapt to toilet
SC7060A	5 in	350lbs/159kg	Inner width range: \geq 200mm, inner length range: \geq 230mm
SC7060B	5 in	300lbs/136kg	

NOT SUITABLE for toilets with grooves or inclined surfaces from the inside to the outside.

Please check the size of your toilet before choosing the raised toilet seat.

PACKAGE LIST

No.	Name	Picture	SC706 0A	SC706 0B
1	Raised Toilet Seat		1	1
2	Armrest		0	2
3	Manual		1	1

CAUTIONS

1. Always be sure that the seat is correctly and securely locked in place before using.
2. Be sure that the locking of the seat is tight at all times, thus securing the seat to the toilet bowl. Physically challenged users should be supervised or assisted whenever using this product.
3. The raised toilet seat **MUST NOT** be used when overloaded. User's weight should be centered over toilet seat, not too forward or too far to one side, as unit may tip.
4. This product is designed for use with the most common types of commodes. It will not fit all commode types; be sure that the unit is correctly installed, fits properly and is stable before use.
5. For models with armrests: The arms are for the purpose of providing assistance when getting on and off the seat. **DO NOT** try to use the arms to

support full body weight during transfer, as this may result in failure of the unit and risk of injury.

6. If you have a style of commode that this product will not fit properly, or cannot be tightened securely as noted above, it is recommended that you discontinue usage.

7. Regularly check for secure assembly and contact support if in doubt. Ensure the product's legs are even and locked before use, and discontinue usage if there are signs of wear, damage, or missing parts. Do not disassemble if not necessary, as it might compromise the longevity of the product. Do not modify the product unless specified in the manual.

8. Exercise caution during operation and assembly. Avoid extreme temperatures or open flames. If an allergic reaction occurs, cease use immediately and consult a healthcare professional.

9. Before use, confirm with your healthcare provider that the product meets your specific needs. The following guidelines are not a substitute for medical advice.


INSTRUCTIONS

●To install the armrests(Neglect this step for the products without armrests)

1. To install arms, depress the brass push buttons and insert the arms into the telescoping tubes on the sides of the seat. Push down until buttons engage and lock into the toilet seat. Arms should curve outward, away from center of toilet. and a "click" sound.

2. Repeat the process for the rest of the holes.

3. To remove arms, depress the brass push buttons and pull the arms away from the toilet seat.

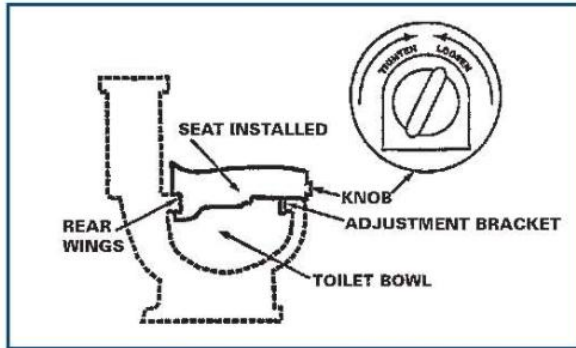●To install the Raised Toilet Seat

1. Set the Raised Toilet Seat onto the toilet. Ensure inner portion of the anchor insert of the Raised Toilet Seat is placed inside the toilet bowl and sets against the back of the toilet rim.
2. Tighten the clamp on the unit by turning the gray knob in the front clockwise until it is securely locked onto the toilet.

NOTE: DO NOT over tighten the knob. This may result in the plastic clamp breaking.

3. The square clamp on the bottom of the Raised Toilet Seat should be facing toward the inside of the toilet's front rim. The clamp may have to be loosened, by turning the gray knob counterclockwise, to fit inside the toilet.
4. Pull up on the Raised Toilet Seat to make sure it is securely locked onto the toilet. Always apply even pressure around the Raised Toilet Seat.

MAINTENANCE

1. REGULAR CLEANING IS RECOMMENDED, USE A NON-ABRASIVE CLEANER OR DETERGENT. RINSE WITH WARM WATER AND DRY COMPLETELY. ABRASIVE CLOTHS MAY DAMAGE THE SURFACE.
2. PLEASE REGULARLY INSPECT YOUR RAISED TOILET SEAT WITH ARMS AND ITS PARTS FOR ANY CORROSION, BREAKAGE, COMPRESSION OR WEAR.

Manufacturer: FOSHAN SUNCARE MEDICAL PRODUCTS CO., LTD.
Address: 2-3 Floor of 65 Building, Zhongnan Industry Garden, No.88
Haitian Road, Gaoming District, Foshan City, Guangdong, 528500, CHINA.

Imported to AUS: SIHAO PTY LTD. 1 ROKEVA STREETEASTWOOD
NSW 2122 Australia

Imported to USA: Sanven Technology Ltd. Suite 250, 9166 Anaheim
Place, Rancho Cucamonga, CA 91730

UK	REP
-----------	------------

YH CONSULTING LIMITED. C/O YH Consulting
Limited Office 147, Centurion House, London
Road, Staines-upon-Thames, Surrey, TW18 4AX

EC	REP
-----------	------------

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Tillverkare: FOSHAN SUNCARE MEDICAL PRODUCTS CO ., LTD .
Importerad till Australien: SIHAO PTY LTD. 1 ROKEVA
Importerad till USA: Sanven Technology Ltd. Suite 250, 9166 Anaheim
E-CrossStu GmbH