VEVOR

Affordable. Reliable. Home Improvement.

Raised Toilet Seat

Model: SC7060D-4A

VEVOR

Affordable. Reliable. Home Improvement.

Raised Toilet Seat

Model: SC7060D-4A

This is the original instruction, please read all manual instructions carefully before operating. VEVOR reserves a clear interpretation of our user manual. The appearance of the product shall be subject to the product you received. Please forgive us that we won't inform you again if there are any technology or software updates on our product.

Thank you very much for choosing this product. Please always handle it with care. Please read all of the instructions before using it. The information will help you achieve the best possible results.

SPECIFICATION

Model	Raised height	load	Adapt to toilet width
SC7060D-4A	4 in	300lbs/136kg	310-390mm

- Adjustable range of clamping handles on both sides is 310-390mm.
- Please check the size of your toilet before choosing the raised toilet seat.

- Check if the outer wall of the toilet is vertical, the slope may not clamp tightly.

PACKAGE LIST

No.	Name	Picture	SC7060 D-4A
1	Raised Toilet Seat with cover		1
2	Clamp		2
3	Armrest		2
4	Manual		1

CAUTIONS

1. Always be sure that the seat is correctly and securely locked in place before using.
2. Be sure that the locking of the seat is tight at all times, thus securing the seat to the toilet bowl. Physically challenged users should be supervised or assisted whenever using this product.
3. The raised toilet seat **MUST NOT** be used when overloaded. User's

weight should be centered over toilet seat, not too forward or too far to one side, as unit may tip.

4. This product is designed for use with the most common types of commodes. It will not fit all commode types; be sure that the unit is correctly installed, fits properly and is stable before use.

5. For models with armrests: The arms are for the purpose of providing assistance when getting on and off the seat. DO NOT try to use the arms to support full body weight during transfer, as this may result in failure of the unit and risk of injury.

6. If you have a style of commode that this product will not fit properly, or cannot be tightened securely as noted above, it is recommended that you discontinue usage.

7. Regularly check for secure assembly and contact support if in doubt. Ensure the product's legs are even and locked before use, and discontinue usage if there are signs of wear, damage, or missing parts. Do not disassemble if not necessary, as it might compromise the longevity of the product. Do not modify the product unless specified in the manual.

8. Exercise caution during operation and assembly. Avoid extreme temperatures or open flames. If an allergic reaction occurs, cease use immediately and consult a healthcare professional.

9. Before use, confirm with your healthcare provider that the product meets your specific needs. The following guidelines are not a substitute for medical advice.

INSTRUCTIONS

1. Place the toilet riser face down on a flat surface and rotate the brackets into the appropriate side locations.
2. Place each armrest into the holes in the bottom of the toilet seat. The metal buttons should be facing out.
3. Slide the armrest tube into position until the button clicks into the hole and a "click" sounds. Repeat the process for the rest of the holes.
4. Insert the side clamps.

5. set the toilet riser. Make sure the inside edge is on the inside of the toilet and the front edge is on the outside (B).

6. Turn the head of each of the two side supports clockwise. Check that the riser is fixed and immovable on the toilet. The riser should fit in a perfectly flat position.

7. Once the toilet riser is in place and fixed, you can raise and lower the lid when you see fit for greater hygiene.

MAINTENANCE

1. REGULAR CLEANING IS RECOMMENDED, USE A NON-ABRASIVE CLEANER OR DETERGENT. RINSE WITH WARM WATER AND DRY COMPLETELY. ABRASIVE CLOTHS MAY DAMAGE THE SURFACE.
2. PLEASE REGULARLY INSPECT YOUR RAISED TOILET SEAT WITH

ARMS AND ITS PARTS FOR ANY CORROSION, BREAKAGE,
COMPRESSION OR WEAR.

Manufacturer: FOSHAN SUNCARE MEDICAL PRODUCTS CO., LTD.
Address: 2-3 Floor of 65 Building, Zhongnan Industry Garden, No.88
Haitian Road, Gaoming District, Foshan City, Guangdong, 528500, CHINA.

Imported to AUS: SIHAO PTY LTD. 1 ROKEVA STREETEASTWOOD
NSW 2122 Australia

Imported to USA: Sanven Technology Ltd. Suite 250, 9166 Anaheim
Place, Rancho Cucamonga, CA 91730

UK	REP
-----------	------------

YH CONSULTING LIMITED. C/O YH Consulting
Limited Office 147, Centurion House, London
Road, Staines-upon-Thames, Surrey, TW18 4AX

EC	REP
-----------	------------

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Tillverkare: FOSHAN SUNCARE MEDICAL PRODUCTS CO ., LTD .
Importerad till Australien: SIHAO PTY LTD. 1 ROKEVA
Importerad till USA: Sanven Technology Ltd. Suite 250, 9166 Anaheim
E-CrossStu GmbH