# **VEVOR<sup>®</sup>**

## **TOUGH TOOLS, HALF PRICE**

Technical Support and E-Warranty Certificate

[www.vevor.com/support](http://www.vevor.com/support)

### **ROLLING WALKER**

**MODEL:FBL427113**

We continue to be committed to provide you tools with competitive price. "Save Half", "Half Price" or any other similar expressions used by us only represents an estimate of savings you might benefit from buying certain tools with us compared to the major top brands and does not necessarily mean to cover all categories of tools offered by us. You are kindly reminded to verify carefully when you are placing an order with us if you are actually saving half in comparison with the top major brands.

# VEVOR<sup>®</sup>

TOUGH TOOLS, HALF PRICE

ROLLING WALKER

MODEL: FBL427113

*The product color is for reference only and is mainly promoted on the website*

## NEED HELP? CONTACT US!

Have product questions? Need technical support? Please feel free to contact us:

**Technical Support and E-Warranty Certificate**  
**[www.vevor.com/support](http://www.vevor.com/support)**

This is the original instruction, please read all manual instructions carefully before operating. VEVOR reserves a clear interpretation of our user manual. The appearance of the product shall be subject to the product you received. Please forgive us that we won't inform you again if there are any technology or software updates on our product.

# INSTRUCTIONS

## **Preface**

First of all, let us express our heartfelt thanks to our customers who have purchased our equipment. This manual is for customer service to make better use of this equipment. We hope the company's products and services to bring you the canal courtesy.

## **Pay attention**

This product must be used in strict accordance with the use method of this manual, misuse of any other methods caused by misuse. The Company does not give any guarantee of any kind, including (but not limited to) the marketability offered to the Company for a particular purpose. Implied warranties with fitness.

The company accepts no liability. The Company shall not be liable for any errors contained in this material, or for any accidental or consequential damages resulting from the provision, actual performance or use of this manual.

This manual contains proprietary information protected by patent law. All rights reserved. Without the written consent of the Company, this manual is not allowed to be photocopied, copied, or translated any part into another language.

The content contained in the manual may be changed without the user's notice.

## **Manufacturer's liability**

The Company shall be held responsible for the safety and reliability of the performance of the equipment only if the assembly operation, returning, improvement and maintenance are carried out by personnel approved by the Company, the electrical equipment concerned conforms to national standards, and the equipment is used in accordance with the operating instructions.

## **Guarantee**

Manufacturing process and raw materials: The Company guarantees that the Equipment (except parts) are in regular use and repair condition, free of

manufacturing process and raw materials faults for one year from the date of shipment. None of our inspections is perfect.

## **PRECAUTIONS**

### **Precautions for use**

#### **Warning:**

1. If the equipment feels loose, falls or is damaged, please contact the company in time;
2. Keep work area clean and well lit. Cluttered or dark areas invite accidents.
- 3.Keep children and bystanders away while operating, distractions will cause you to lose control.
- 4.During normal use, please use with the accompaniment of family or friends.
- 5.Reasonably use according to one's own situation, and do not overuse when the body is not in a timely manner.
- 6.BEFORE USE, PLEASE CONFIRM WHETHER THE COMPONENTS ARE COMPLETE, WHETHER THE BRAKE IS SENSITIVE, AND ASSEMBLE THE ENTIRE PRODUCT IN STRICT ACCORDANCE WITH THE INSTRUCTIONS.
- 7.CLEANING PRODUCTS NEED TO BE WASHED WITH WATER AND DRY IN TIME. IF THE SURFACE COATING OF THE PRODUCT HAS SCRATCHES AND FALLS OFF, REPAIR MATERIALS NEED TO BE USED IN TIME TO ACHIEVE THE BEST ANTI-RUST EFFECT.
- 8.PLEASE ENSURE THAT THE REAR WHEEL OF THE PRODUCT IS LOCKED TO AVOID SLIDING WHEN THE USER STAYS IN THE SLOPING ENVIRONMENT.
- 9.PLEASE ENSURE THAT THE FOLDING MECHANISM OF THE SEAT PLATE IS LOCKED WHEN USING THIS PRODUCT.

## **SAVE THESE INSTRUCTIONS**

## PRODUCT PARAMETERS

Product Name	Rolling Walker
model	FBL427113
colour	Blue/Black/Red
Wheel size	Front 8in/Back 10in
Load bearing capacity/lb	300

## INSTALLATION STEPS

1.As shown in the figure: Place the front wheel into the main frame pipe bend, align it with the hole position, and lock it with screws and knobs.

2. As shown in the figure: Install the front wheel on the other side as shown above.

3. As shown in the figure: Place the rear wheel into the straight tube of the main frame, align it with the hole and push it inward until the marble pops out.

4. As shown in the figure: Install the front wheel on the other side as shown above.

5. As shown in the figure: Unfold the main frame and press the two sides of the seat plate by hand to make the hook catch the round pin. When retracting, pull the seat plate webbing up by hand to complete the retraction process.

6. As shown in the figure: Install the left and right armrest tubes into the

main frame tube until the marble pops out.

7. As shown in the figure: Install the right foot pedal assembly into the pin on the right side of the main frame, rotate it to the left to fix the foot pedal assembly.

8. As shown in the figure: Install the left foot pedal assembly into the pin on the right side of the main frame, rotate it to the right to fix the foot pedal assembly.

9. As shown in the figure: Install the crutch holder into the crutch holder.

10. Assembly completed rendering.

## **About this item**

1. **Stable & Not Easy to Tip Over:** 8" front rear wheels, 360° swivel front wheels for easy maneuvering, sturdy enough for indoor and outdoor smooth ride.
2. **Convenient to Go Outside:** Zippered carrying pouch, keep all your important items instantly within reach, free your hands to fully enjoy your life.
3. **Nice Service:** 100% tested before leaving the factory and 24-hour friendly email customer service.
4. This product comes with self braking and auxiliary seat cushion functions. When the operator needs to rest, they can turn on the self braking and sit on the seat cushion to rest.
5. There is a portable storage bag under the product cushion, which is convenient for storing some miscellaneous items and purchasing some daily necessities for storage.

**Manufacturer:** Shanghaimuxinmuyeyouxiangongsi

**Address:** Baoshanqu Shuangchenglu 803long 11hao 1602A-1609shi Shanghai

**Imported to AUS:** SIHAO PTY LTD, 1 ROKEVA STREETEASTWOOD NSW 2122  
Australia

**Imported to USA:** Sanven Technology Ltd.

Suite 250,9166 Anaheim Place, Rancho Cucamonga, CA 91730

SHUNSHUN GmbH

Römeräcker 9 Z2021,76351

Linkenheim-Hochstetten, Germany

Pooledas Group Ltd

Unit 5 Albert Edward House, The

Pavilions Preston, United Kingdom

**VEVOR<sup>®</sup>**

**TOUGH TOOLS, HALF PRICE**

**Technical Support and E-Warranty Certificate**

**[www.vevor.com/support](http://www.vevor.com/support)**

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**Importato in AUS:** SIHAO PTY LTD, 1 ROKEVA STREETEASTWOOD NSW
**Importato negli Stati Uniti:** Sanven Technology Ltd.