VEVOR

Affordable. Reliable. Home Improvement.

Tool Sanitizer

Technical Support and E-Warranty Certificate
www.vevor.com/support

VEVOR

Affordable. Reliable. Home Improvement.

Tool Sanitizer

Model: CH-360T

This is the original instruction, please read all manual instructions carefully before operating. VEVOR reserves a clear interpretation of our user manual. The appearance of the product shall be subject to the product you received. Please forgive us that we won't inform you again if there are any technology or software updates on our product.

Symbol	Symbol Description
	<p>Warning: To reduce the risk of injury, the user must read the instructions manual carefully.</p>
	<p>This symbol, placed before a safety comment, indicates a kind of precaution, warning, or danger. Ignoring this warning may lead to an accident. To reduce the risk of injury, fire, or electrocution, please always follow the recommendation shown below.</p>
	<p>CORRECT DISPOSAL:</p> <p>This product is subject to the provision of European Directive 2012/ 19/EC. The symbol showing a wheeled bin crossed through indicates that the product requires separate refuse collection in the European Union. This applies to the product and all accessories marked with this symbol. Products marked as such may not be discarded with normal domestic waste, but must be taken to a collection point for recycling electrical and electronic devices.</p>

FCC INFORMATION

CAUTION: Changes or modifications not expressly approved by the party responsible for compliance could void the user's authority to operate the equipment!

This device complies with Part 15 of the FCC Rules. Operation is subject to the following two conditions:

- 1) This product may cause harmful interference.
- 2) This product must accept any interference received, including interference that may cause undesired operation.

WARNING: Changes or modifications to this product not expressly approved by the party responsible for compliance could void the user's authority to operate the product.

SAFETY INSTRUCTIONS


WARNING: Do not expose eyes or skin to UV light.

- 1.The first time you use it, there will be white smoke coming out. This is a normal phenomenon. The white smoke will significantly decrease afterwards.
- 2.When using the sterilizer for the first time,operate it in an empty load mode for working cycle,then open the door for 20 minutes.This operation can eliminate any odd smell inside the cabinet.
- 3.Make sure to drain all water the utensils before placing them in the sterilizer.The utensils should not be stacked but placed vertically one by one to achieve the best sterilization effect.
- 4.In case of any abnormal conditions(such as unusual noise,odd smell,or smoke),immediately unplug the power cord of the sterilizer and contact the repair center or the manufacturer for inspection and repair.Do not use it again until the cause of the fault is resolved.
- 5.When maintaining or cleaning the device,make sure to unplug the power cord to ensure complete disconnection from the power source.
- 6.Wait for 10 minutes after the sterilization cycle completed before opening the door.Use the utensils only after they have cooled down to avoid affecting the sterilization effect or causing burns.
- 7.Do not install sterilizer near flammable materials.
- 8.If the power cord is damaged,it must be replaced by a professional from the manufacturer,its repair department,or similar department to avoid danger.
- 9.Do not place the sterilizer in areas prone to moisture or water splashes,and do not use heating appliances near the sterilizer,as this may cause deformation or malfunction of the sterilizer.
- 10.After use,if it will not be used for a long,please cut off the power and do not leave the power cord in a connected state to avoid danger.
- 11.It is strictly prohibited to directly rinse the with water or immerse the machine in water.

MODEL AND PARAMETERS

Model	CH-360T
Input	AC 110V 60Hz(US), AC 220-240V 50Hz(EU&AU)
Max. Power	300W
Temperature setting	0~220℃
Time setting	5-60 min
Size	349*193*175mm

STRUCTURE DIAGRAM

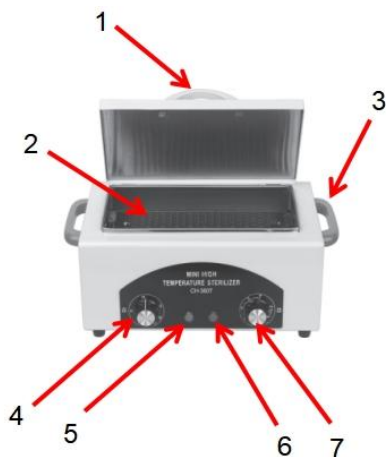8

1.Handle	2.Tray
3.Handle	4.Temperature adjustment knob
5.Power indicator light	6.Work indicating lamp
7.Time adjustment knob	8.Electrical outlets

PARTS LIST

	
<p style="text-align: center;">Tool Sanitizer x1</p>	<p style="text-align: center;">Tray x1</p>
	
<p style="text-align: center;">Power Cord x1</p>	<p style="text-align: center;">Ring x2 Fuse x1</p>
<div style="text-align: center;">  <p>VEVOR 10000 INDUSTRIAL ROAD, PHOENIX Special Report: A-52 Meeting Customer's Demand ELECTRIC MELTING FURNACE MODEL: V2-20041</p> </div>	
<p style="text-align: center;">User Manual x1</p>	

OPERATION METHOD

1. Please remove the protective film on the tray for the first use.
2. Plug the matching power source we provide.
3. Put your instrument to be sterilized on the tray and close the cover.
4. Adjust the time dial according to your requirement, activating the device. The red indicator will light on which means the device is under working mode. Next, adjust the temperature dial according to your requirement, and by now the green indicator will light on. Both of the lights will go out when the device has finished its work. Please note that neither of the indicator will light on if you only adjust the temperature dial. Generally speaking, all instrument and goods can be totally sterilized and disinfected in 15 minutes at 125 Celsius degrees.
5. Adjust the temperature by dialing the left hand knob.

6. Unplug the power supply after each use.
7. After sterilization, hook the ring onto the tray and fetch it out the device.

Maintenance

1. Make sure to unplug the power cord before cleaning or maintaining the disinfection cabinet.
2. Wear gloves cleaning or maintenance to prevent cuts from sharp edges inside the cabinet.
3. Regularly wipe off stains from the surface to keep the cabinet clean and attractive.
4. Electrical components must not be immersed in water. Do not rinse the cabinet with water.
5. Regular maintenance is recommended. Unplug the power cord, use a cloth with a small amount of neutral detergent to gently remove stains inside and outside the cabinet, then use a clean damp cloth to remove the detergent.
6. When disinfection cabinet is not in use, it should be unplugged. If it is to be unused for a long time, moved, or serviced, make sure to unplug the power cord first.
7. If any abnormality occurs during use, stop using it immediately and unplug the power cord. To avoid danger, installation or repair must be done by a qualified electrical repair person.

Common Fault Guidelines

Fault Description	Cause Analysis	Solution
The device is not powered on	The power cord is not plugged in	Plug the matching power source we provide
The device is not functioning	The time adjustment knob is not adjusted	Adjust the time adjustment knob

Manufacturer: Foshan Kangzhiqing Electrical Appliance Co.,Ltd

Address: No.23-2,Liuyi San Road,Nanhai District,Foshan City,Guangdong Province, China

Imported to AUS: SIHAO PTY LTD. 1 ROKEVA STREETEASTWOOD NSW 2122 Australia

Imported to USA: Sanven Technology Ltd. Suite 250, 9166 Anaheim Place, Rancho Cucamonga, CA 91730

UK	REP
-----------	------------

YH CONSULTING LIMITED. C/O YH Consulting Limited Office 147, Centurion House, London Road, Staines-upon-Thames, Surrey, TW18 4AX

EC	REP
-----------	------------

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Fabricante: Foshan Kangzhiqing Electrical Appliance Co., Ltd. Dirección: No.23-2,
Importado a EE. UU.: Sanven Technology Ltd. Suite 250, 9166 Anaheim Place, Rancho
YH CONSULTING LIMITADA. A/C YH Consulting Limited
E-CrossStu GmbH