MODEL:PZHXQ-F01

MODEL:SGHXQ-C01

(The picture is for reference only; please refer to the actual object)

This is the original instruction, please read all manual instructions carefully before operating. VEVOR reserves a clear interpretation of our user manual. The appearance of the product shall be subject to the product you received. Please forgive us that we won't inform you again if there are any technology or software updates on our product.

Thank you very much for choosing this Choking Rescue Device. Please read all of the instructions before using it. The information will help you achieve the best possible results.

This product uses mechanical action to create one-way suction and is designed with a one-way valve to prevent additional injury. It's made of high-quality silicone material, is portable and durable, reusable, and doesn't need replacement parts. It's not intended for medical use, but it can be used in emergency situations and is easy to use with clear instructions.

MODEL:PZHXQ-F01

*If the victim is unconscious, carefully place it on a flat and sturdy surface.

*Choose a mask size that is suitable for the victim; If used on a child, please make sure to use children's sizes.

*Place the mask on the mouth and nose, ensuring a tight seal to generate effective suction.

EASY TO USE IN CHOKING EMERGENCIES

PLACE


Place the mask over the victim's nose and mouth. Hold the mask with your hand so that it forms a good all-around seal on the victim's face.

PUSH

Push the plunger down with your strong hand, all air will escape out of the one-way valve system preventing an obstruction from being pushed further into the victim.

PULL

Pull back on the plunger while holding the mask place. This will create suction. If the obstruction does not come out on the first pull, continue to push, and pull the plunger without removing the device from the victim's face until the obstruction is removed.

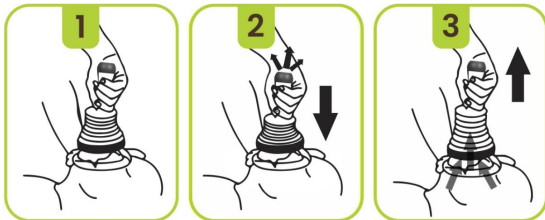MODEL:SGHXQ-C01

*If the victim is unconscious, carefully place it on a flat and sturdy surface.

*Choose a mask size that is suitable for the victim; If used on a child, please make sure to use children's sizes.

*Place the mask on the mouth and nose, ensuring a tight seal to generate effective suction.

EASY TO USE IN CHOKING EMERGENCIES

PLACE

Place the mask over the victim's nose and mouth.

PRESS

Press and hold the holder&vent valve with your hand so that it forms a good all-around seal on the victim's face. This allows it to generate strong suction.

PULL

Pull back on the plunger while holding the mask place. This will create suction. If the obstruction does not come out on the first pull, continue to push, and pull the handle&plunger without removing the device from the victim's face until the obstruction is removed.

Manufacturer: Yiwu Daqida Plastic Products Co., Ltd.

Address: 1st Floor, Building 1, No. 7 Yongle Road, Niansanli Street Yiwu, Zhejiang CN 322000

Imported to AUS: SIHAO PTY LTD. 1 ROKEVA STREET EASTWOOD NSW 2122 Australia

Imported to USA: Sanven Technology Ltd. Suite 250, 9166 Anaheim Place, Rancho Cucamonga, CA 91730

YH CONSULTING LIMITED. C/O YH Consulting Limited Office 147, Centurion House, London Road, Staines-upon-Thames, Surrey, TW18 4AX

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Technical Support and E-Warranty Certificate
www.vevor.com/support

VEVOR

Affordable. Reliable. Home Improvement.

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Hersteller: Yiwu Daqida Plastic Products Co., Ltd.
Importiert nach AUS: SIHAO PTY LTD. 1 ROKEVA STREET EASTWOOD
Importiert in die USA: Sanven Technology Ltd. Suite 250, 9166 Anaheim
E-CrossStu GmbH