VEVOR

Affordable. Reliable. Home Improvement.

PET WHEELCHAIR

MODEL: SS3-01 SS3-02

VEVOR

Affordable. Reliable. Home Improvement.

PET WHEELCHAIR

SS3-01

SS3-02

This is the original instruction, please read all manual instructions carefully before operating. VEVOR reserves a clear interpretation of our user manual. The appearance of the product shall be subject to the product you received. Please forgive us that we won't inform you again if there are any technology or software updates on our product.

SPECIFICATIONS

Model	Size
SS3-01	M
SS3-02	L

Pet Wheelchair User Guide

According to the after-sales feedback from users, most pets take about 1 month to fully get familiar with, adapt to and use wheelchairs.

How to Help Your Pet Get the Most Out of Their Wheelchair

Environmental Adaptation

After you have assembled the wheelchair, bring it in your pet's favorite activity area. Leave the wheelchair there and let your pet check it out and get comfortable being around it (i.e. sniffing, touching).

Keep the area quiet and don't let other pets or distractions come into the area.

Harness Training

Let your pet get used to the front/rear leg straps without putting the straps on the wheelchair. Let them try the front and rear separately at first, once they are ok with wearing the straps separately try putting them in the whole harness without the wheelchair. Remember to offer treats or other rewards whenever they show comfort and familiarity to the straps

Conduct 3-5 short wearing sessions daily, each not exceeding 10 minutes. Connect the wheelchair frame only after your pet shows no resistance to wearing the whole harness.

Phased Training Steps

Stage 1: Building Positive Associations

Immediately reward your pet with high-value treats after placing them in the wheelchair.

Guide your pet to move forward 1-2 steps using handheld treats, rewarding upon completion.

Train 3 times daily, 5 minutes per session.

Stage 2: Short-Distance Movement

Use a leash to assist with directional control.

Practice straight-line walking in an open, obstacle-free area.

Reward every 3 meters of successful movement, gradually increasing the distance between rewards

Stage 3: Complex Environment Adaptation

Introduce slight turns and furniture avoidance training.

Provide manual assistance for the first time passing through doorways or obstacles.

Offer more rewards after completing complex maneuvers.

Key Precautions

Safety Control

Use a short leash (≤ 1 meter) for the first 3 days.

Avoid hazardous areas such as stairs or slopes.

Immediately disconnect the harness and comfort your pet if they show signs of struggle or panic.

Usage Duration Management

Week 1: Single sessions should be no more than 15 minutes, daily total no more than 1 hour.

Week 2: Gradually extend to 30 minutes per session, daily total no more than 2 hours.

Long-term use: Check for limb compression once for every hour of use.

Behavioral Observations

Support your pet's abdomen during urination/defecation.

Check strap tightness if frequent chewing or licking occurs.

If your dog refuses to move, you can try changing the type of treat you are offering.

Components

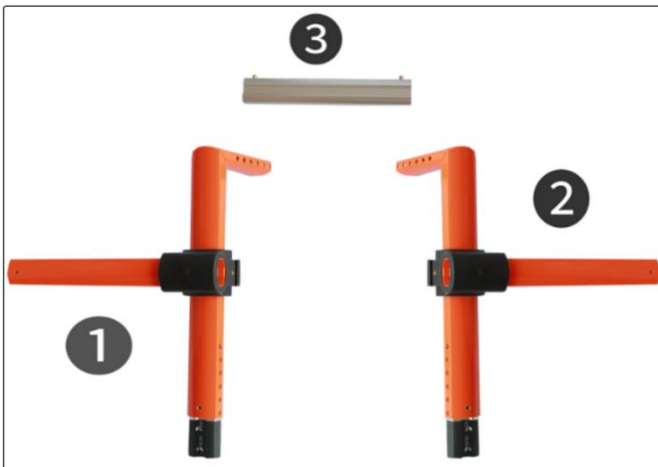
The pictures in the instruction manual are for reference only. Please refer to the actual product for the color.

No.	Parts	Qty.
1	Left Support Frame	x1
2	Right Support Frame	x1
3	Crossbar	x1
4	Left Wheel Rod & Wheel	x1
5	Right Wheel Rod & Wheel	x1

6	Body Length Rod (pre-installed in ❶ ❷)	x2
7	Foam Sleeve	x2
8	L-shaped tool	x1
9	Leash	x1
10	Front Limb Harness	x1
11	Rear Limb Harness	x1
12	Waist Strap	x1
13	M4 screw	x3 (One spare unit)
14	M5 screw	x12 (Two backups)

Installation and length adjustment instructions for the frame

Step 1: Find parts ❶, ❷, and ❸



Step 2: Press the spring-loaded ball down on part ③ and insert into part ②.

Do the same thing for part ①

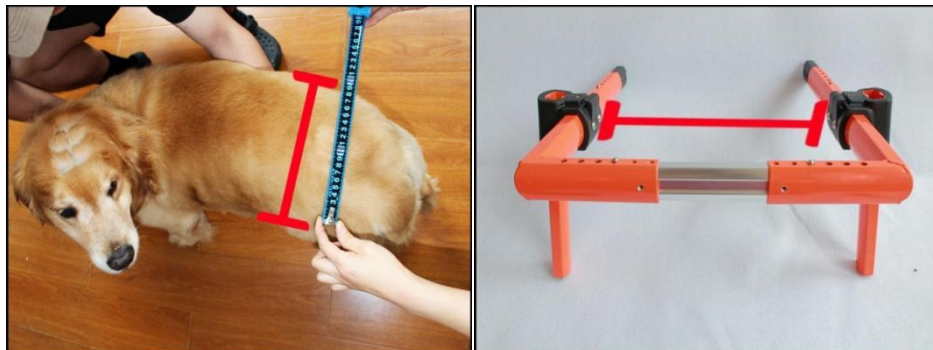
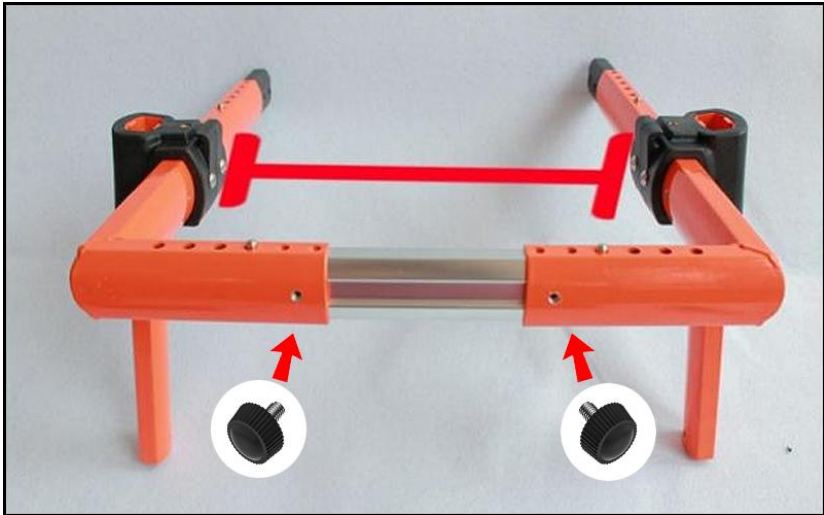
Step 3: Measure your dog's body width. Press in the ball and slide the crossbar until the ball snaps into the appropriate hole (Two M5 screws with serial number 14.) If a measurement falls within the adjustment range of both gears, select the setting for the larger gear. Should any issues arise, consult the FAQ section at the end of the manual.

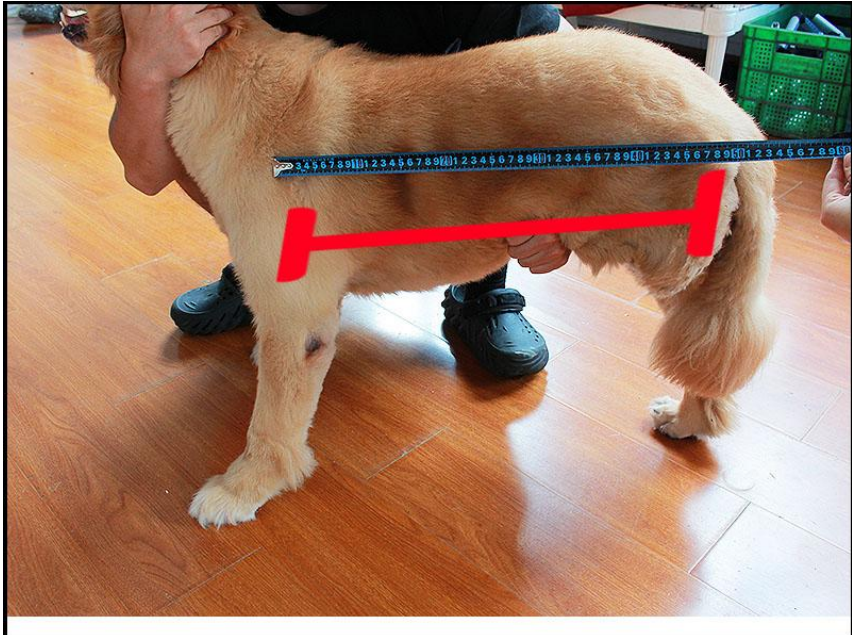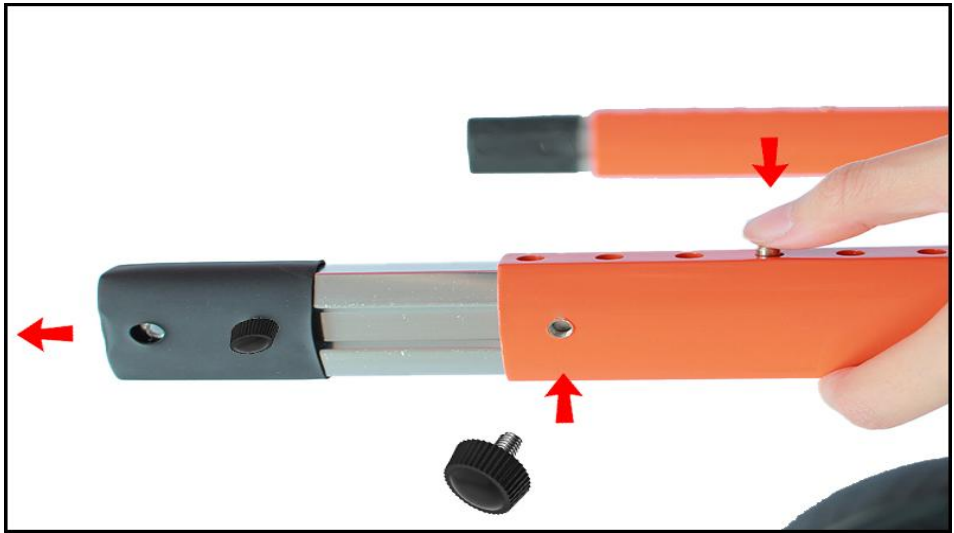Step 4: Measure your dog's body length (top of the sternum(breastbone) to tail base). Press the spring-loaded ball on parts ①② to extend the rod until the ball snaps into position. Insert and tighten the screw as seen below. Repeat on the other side. (Two M5 screws with serial number 14.)


M


L

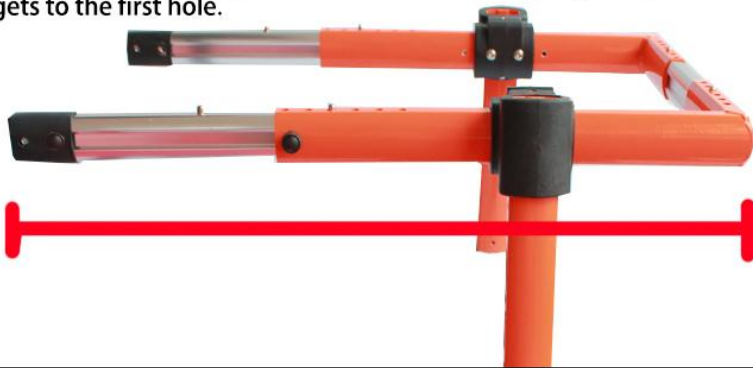Length Extension Rods: Some customers have an issue where they move the first ball from the last hole to the first and stop pulling. Please be aware that you can push the ball down and pull beyond the first hole! The medium size has three balls and the large has 5 balls. You can keep adjusting until the last ball gets to the first hole.



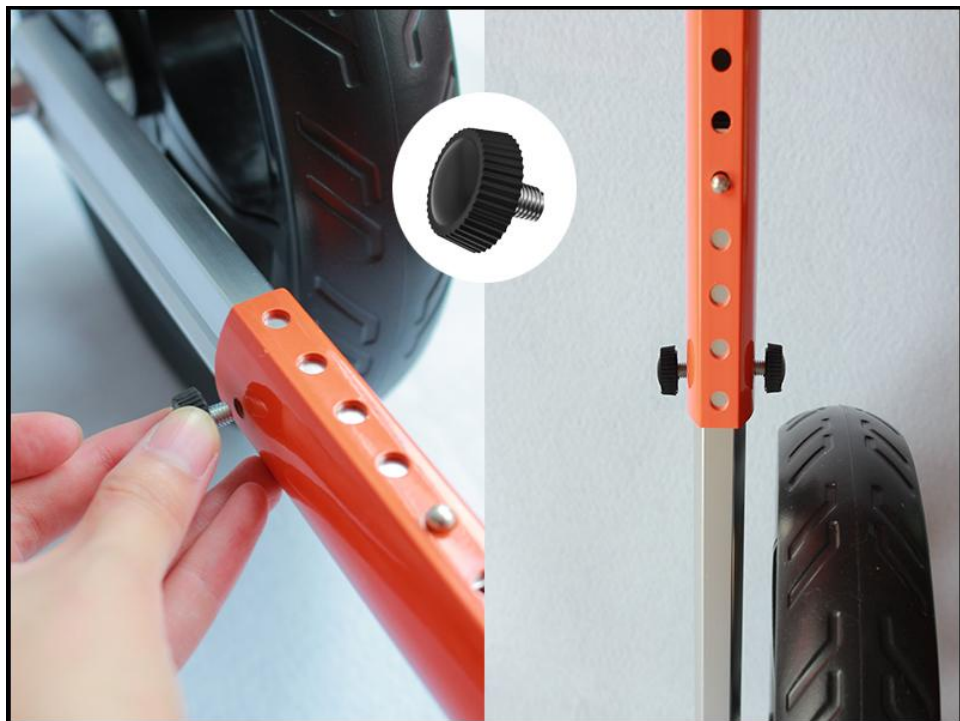
Step 5: Measure your dog's height (ground to sternum). Press the ball down on part ④ and insert into part ① (see pics below for reference). Repeat on the other side. After it's the right height, secure with these screws (See below) . (Four M5 screws with serial number 14.)


Instructions for how to wear and adjust the harness properly

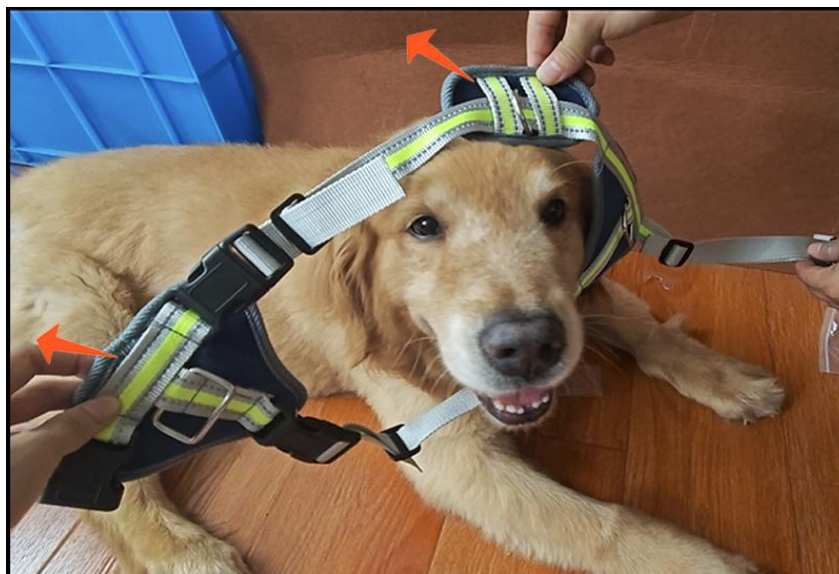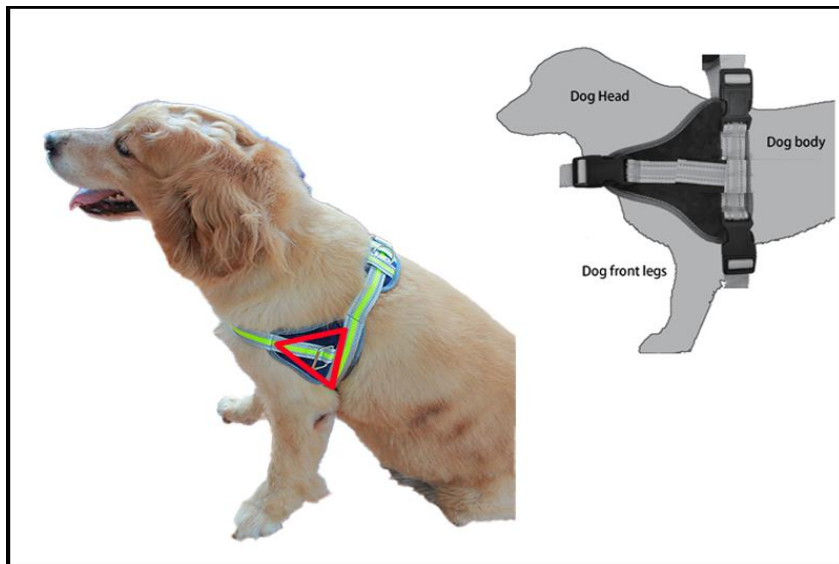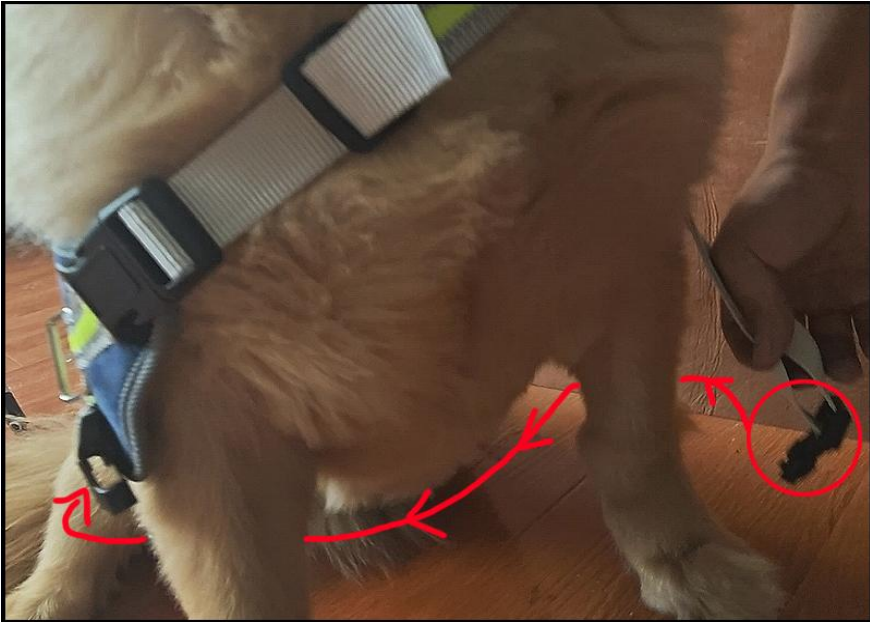Step 1: Align the triangular pieces on the harness⑩ with your dog's front legs. The three-point harness has already been pre connected at two points. As shown in the diagram(see below and next page), it should be fitted onto the dog's head. Please pass the remaining connection through the armpit of the dog's front legs.

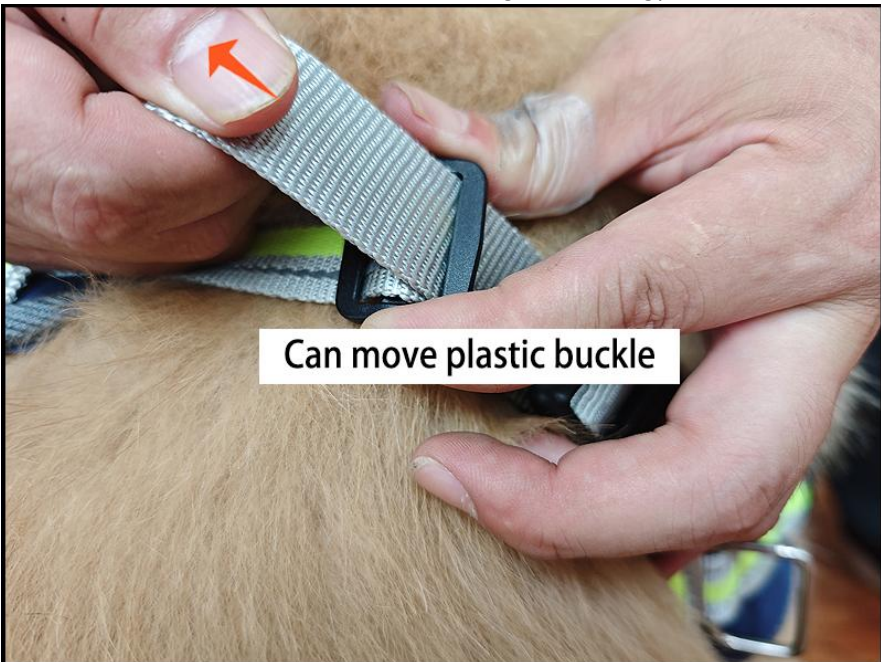Step 2: Adjust reflective strap length: Pull one side of the strap, slide the black plastic piece, and adjust it to the appropriate position (see FAQ at end of manual for more details on fitting and sizing).


Step 3: For the hind leg harness, position the harness¹¹ with the holes aligned to your dog's hind legs.

Before pulling up, loosen off the buckle that will go over your dog's back. Then, pull up the harness and connect the buckle. Adjust the buckle until the harness feels secure. Please note that the harness may not go over all of the dog's legs and back, but this is OK. This harness is designed to hold up the hind legs when it is connected to the wheelchair.

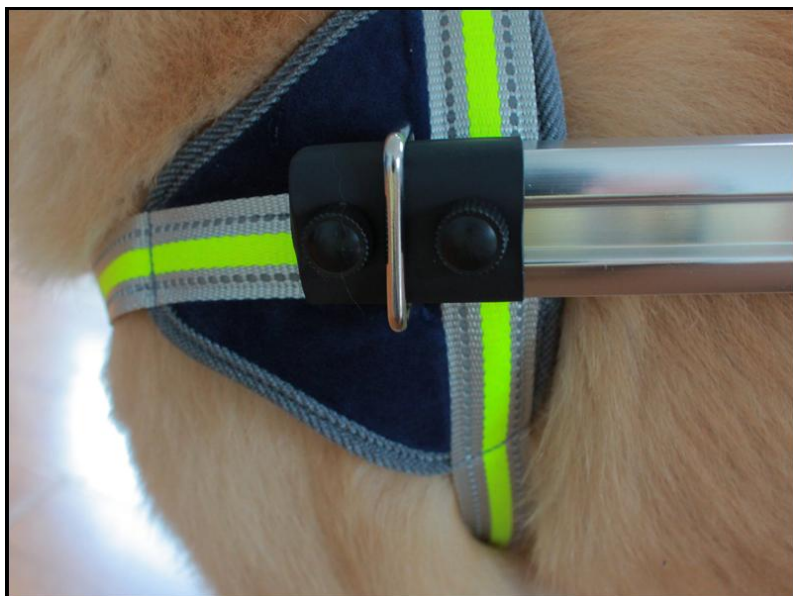
Step 4: Insert the black piece on the front of the frame through the square buckle on front harness⑩. Secure with screws to prevent slippage(see pics below). Repeat on the other side. (Two M5 screws with serial number 14.)

Step 5: Attach the rear harness **11** buckles into the slot on top of the black piece that holds the wheels. (See pics below) Secure it with screws. Repeat on the other side. (Two M4 screws with serial number 13.)

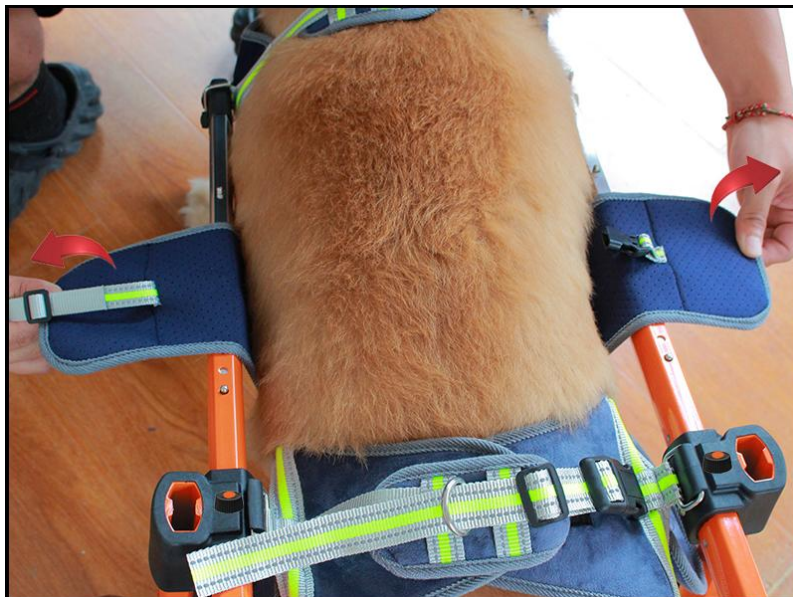Step 6: Position the waist strap 12 under the abdomen. Adjust tightness (snug but comfortable). Wrap the Velcro strap around the frame and attach it . (The belt is used to support and lift up the abdomen, not to wrap all the way around your dog.

Step 4: Install part ⑦ onto the width bar. Place your dog's injured hind leg into foam sleeve ⑦.

This step determines whether to use parts based on the condition of the dog leg. Only use these straps if your dog requires them for support. Not all dogs will need them.

Maintenance and adjustment tips

Daily Checks

Inspect screw tightness and bearing flexibility weekly.

Make sure to hand tighten all screws before use.

Dry the frame after rainy days to prevent rust.

Clean the harness regularly with a pH-neutral detergent.

Size Adjustment

Chest circumference: Allow a one-finger gap for comfort.

Frame height: Ensure leg joints remain naturally bent when standing.

Re-measure your dog every 3 months based on growth/weight changes and make necessary size changes when needed.

FAQs & Troubleshooting

Tips on how to help your dog get used to using its wheelchair

Q: My pet refuses to move or shows anxiety. Is the wheelchair not working?

This is normal during the first few times it tries the wheelchair. Stay patient.

Revisit the "Phased Training Steps" for help on how to encourage your dog to use the

wheelchair.

Use higher-value treats (e.g., fresh meat) during training if normal treats don't seem to help. Avoid forcing usage; keep daily training within the recommended durations.

Q: My pet is moving backwards, not forwards

Check the Chest Strap First! The most common issue causing this issue is the chest strap. . Make sure it sits behind the sternum (breastbone) and before the ribs—never pressing the armpits (causes pain) or belly (prevents pushing). All straps should be snug but not tight, remember that you should be able to easily slide a finger between the strap and your dog.

Q: Why aren't my wheels turning smoothly?

Check the wheels to make sure there is no hair or foreign objects stuck in them

Q: Do I have the correct height?

Your dog's paws must be touching the ground to push naturally. Remember that their joints should have a natural bend to them.

Q: How can I keep my dog from slipping on the floor when learning to use the wheelchair?

Switch locations immediately! Tile/wood floors can cause slipping. Only practice on carpet, grass, or rough ground.

Q: How can I help address fear/resistance to wheelchair use?

Remember to stick to the short acclimation sessions: Start with getting used to the harness first, then try the harness and wheelchair together. Put the wheelchair on multiple times daily—no movement required. Keep the focus on getting comfortable being in it. Reward with treats/praise to build positive associations, even just for small successes.

Q: How can I help my dog learn to walk with the wheelchair?

Guide with treats and always be gentle with them through the whole process.:

Lightly support the belly/hips while slowly nudging them forward.

Lure them forward with a favorite treat/toy held close to the front of their nose.

Instantly reward any forward attempt with praise + treats!

Key Notes:

Never force or punish backward movement! This can worsen fear.

Patience is vital! Progress takes time—even standing calmly is a win.

Q: can I view the parts that will have my dogs legs go through an opening. One review says the openings are too small for her pet. I don't see leg holes in the photo on the description page. where are these leg holes?

These are the two holes in the fabric of the hind limbs.

Q:Can my dog still use her good hind leg to keep its muscle tone? I want this more for her to rest in rather than having to sit down and get back

Yes, your dog can still use her good hind leg to keep its muscle tone.

Q:Will the wheels roll on dirt and uneven ground?

Yes, it can roll on dirt and uneven ground.

Q: Why does a dog refuse to move while in a wheelchair, or lean to the side after taking just a few steps?

It may be because dogs have highly sensitive hearing, and the sound produced by the wheelchair wheels touching the ground during movement is an unfamiliar noise that scares them. In such cases, owners need to comfort their pets and must not force them to walk by pulling. Instead, they can demonstrate the sliding of the wheelchair frame and wheels in front of the pet multiple times to help it get used to the sound.

Sizing Issues

Q: How can I ensure proper wheelchair sizing?

How to measure for length: Measure from the end of the sternum(breastbone) to the base of the tail.

Width adjustment: Measure the widest rib span plus 2 cm allowance.

Height adjustment:

Align the frame with the highest point of your dog's hips.

CAUTION:Avoid positions that are too high (can cause abdominal pressure) or too low (hind legs dragging).

Q: What should I do if the size is incorrect?

When you receive the wheelchair, please note that we've set the length extension rod to its smallest position during packaging. You'll need to install the other width and height adjustments yourself.

Length extension rod adjustment tips: Some customers have had problems where they move the spring loaded ball from the last hole to the first hole and then stop pulling. However, in this case, the length extension rod has only been pulled out one-third of the way.You can continue pulling out the length extension rod to get the correct length for your dog.

Q: How do I adjust the harness?

Tighten the abdominal strap until one finger can slide in.

Ensure the chest strap allows free breathing while still being snug and supportive.

Check strap tightness weekly; adjust if weight changes $\pm 10\%$.

Q: What can I do if parts are lost or damaged?

Confirm the part number using the "Component Coding Diagram" in the manual.

Take clear photos of the damaged/lost part.

Contact customer service personnel through the order page.

Critical Reminders

Immediately stop use and contact support if:

Metal frame shows abnormal deformation/cracks.

Wheel bearings produce metallic friction noises.

Straps exhibit 2 or more consecutive stitching breaks.

Manufacturer: Shanghaimuxinmuyeyouxiangongsi

Address: Shuangchenglu 803nong11hao1602A-1609shi, baoshanqu, shanghai 200000 CN.

Imported to AUS: SIHAO PTY LTD. 1 ROKEVA STREETEASTWOOD NSW 2122 Australia

Imported to USA: Sanven Technology Ltd. Suite 250, 9166 Anaheim Place, Rancho Cucamonga, CA 91730

UK	REP
-----------	------------

YH CONSULTING LIMITED. C/O YH Consulting Limited Office 147, Centurion House, London Road, Staines-upon-Thames, Surrey, TW18 4AX

EC	REP
-----------	------------

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Geïmporteerd naar AUS: SIHAO PTY LTD. 1 ROKEVA STREETEASTWOOD
Geïmporteerd naar de VS: Sanven Technology Ltd. Suite 250, 9166
E-CrossStu GmbH